Comp-Position

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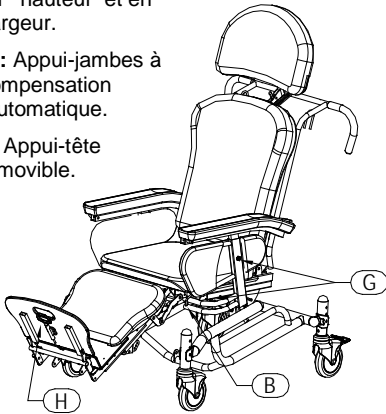
Your Comp-Position positioning chair is built for years of trouble-free service. Carefully read and follow the instructions step-by-step to assemble, operate and care for your chair.

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
Using the *Comp-Position* Chair

For the safest operation, please familiarize yourself with the following components:

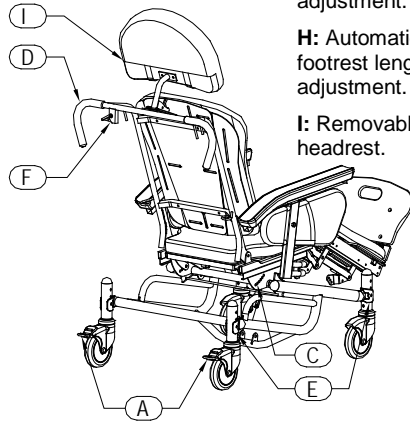
- A :** Swivel wheels with brakes.
- B :** Seat depth adjustment.
- C :** Backrest angle adjustment.
- D :** Push bar.
- E :** Seat height adjustment.
- F :** Tilt-In-Space lever activator.

- G :** Armrest width and height adjustment.
- H :** Automatic footrest length adjustment.
- I :** Removable headrest.

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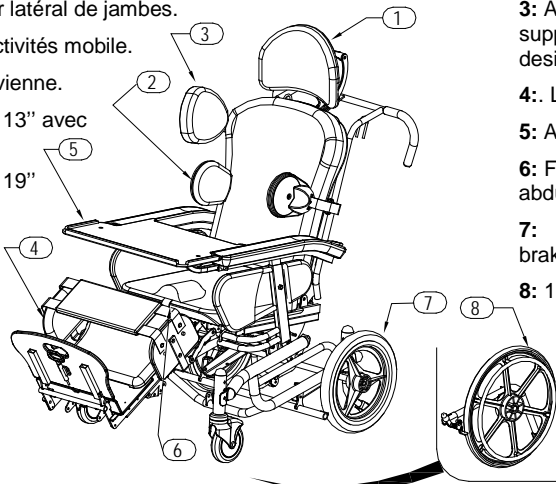Accessories

- 1 :** Headrest Adjustment.
- 2 :** Structural Lateral Supports adjusts to provide comfortable, correct trunk positioning. Removable (if desired).
- 3 :** Adjustable shoulder supports. Removable (if desired).
- 4 :** Lateral legs protector.
- 5 :** Activity Tray.
- 6 :** Fold-away perineal abductor pad.
- 7 :** 13" rear wheels with brake.
- 8 :** 19" rear wheels with brake.

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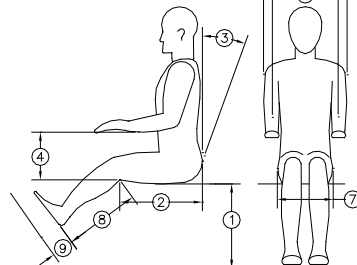
WARNING

All brakes **MUST BE ENGAGED AT ALL TIMES** except during transportation.

This product is equipped with a mechanized legrest. **DO NOT** let children sit on this chair without supervision. **DO NOT** let children operate any mechanism. A child could be seriously injured when declining the legrest. **ALWAYS** leave an empty chair in the upright position.

Keep hands and feet clear of all mechanisms.

The chair should be operated by a healthcare professional.

Be sure all adjustments are correct and bolts and nuts are tightened before using the chair.

All adjustments must be supervised by a healthcare professional.

This product is intended for use in a secure environment and under constant supervision.

Comp-Position Maintenance

For optimum performance, it is necessary that the chair be kept clean and all parts adjusted properly.

When needed, the upholstered surfaces may be washed with a mild detergent diluted or dissolved in water. If necessary, a mild, clear disinfectant can be used for hygienic purposes.

Metal parts of the chair should also be kept clean using the same mild detergent solution as described above.

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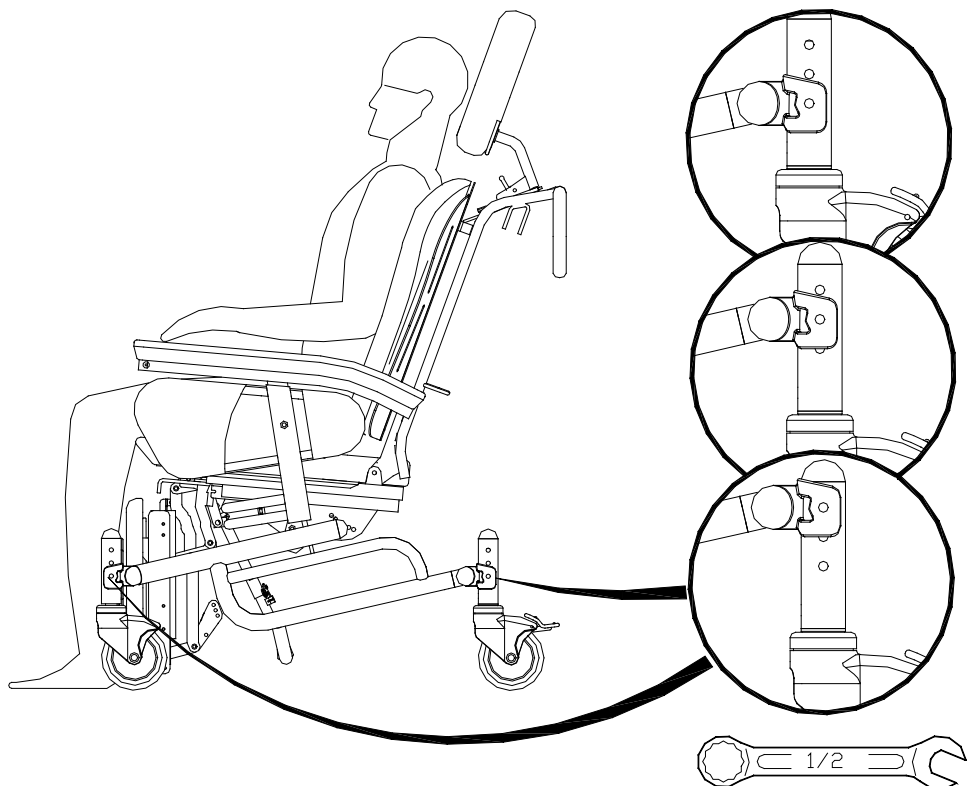
How to use of your *Comp-Position Chair*

All brakes MUST BE ENGAGED AT ALL TIMES, except during transportation.

1. Seat height adjustment.

- 1.1. When the chair is unoccupied, determine if the seat height should be adjusted up or down.
- 1.2. Unscrew and remove the 4 bolts on the seat height adjustment wheel structure as shown below.
- 1.3. Move the seat height adjustment wheel structure to the desired position. BE SURE EACH SIDE IS AT THE SAME LEVEL, tighten the 4 bolts securely with the lock washers in place.
- 1.4. Place the chair in an upright position and check if the seat is correctly adjusted for the occupant, correct if required.

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
2. Seat depth.

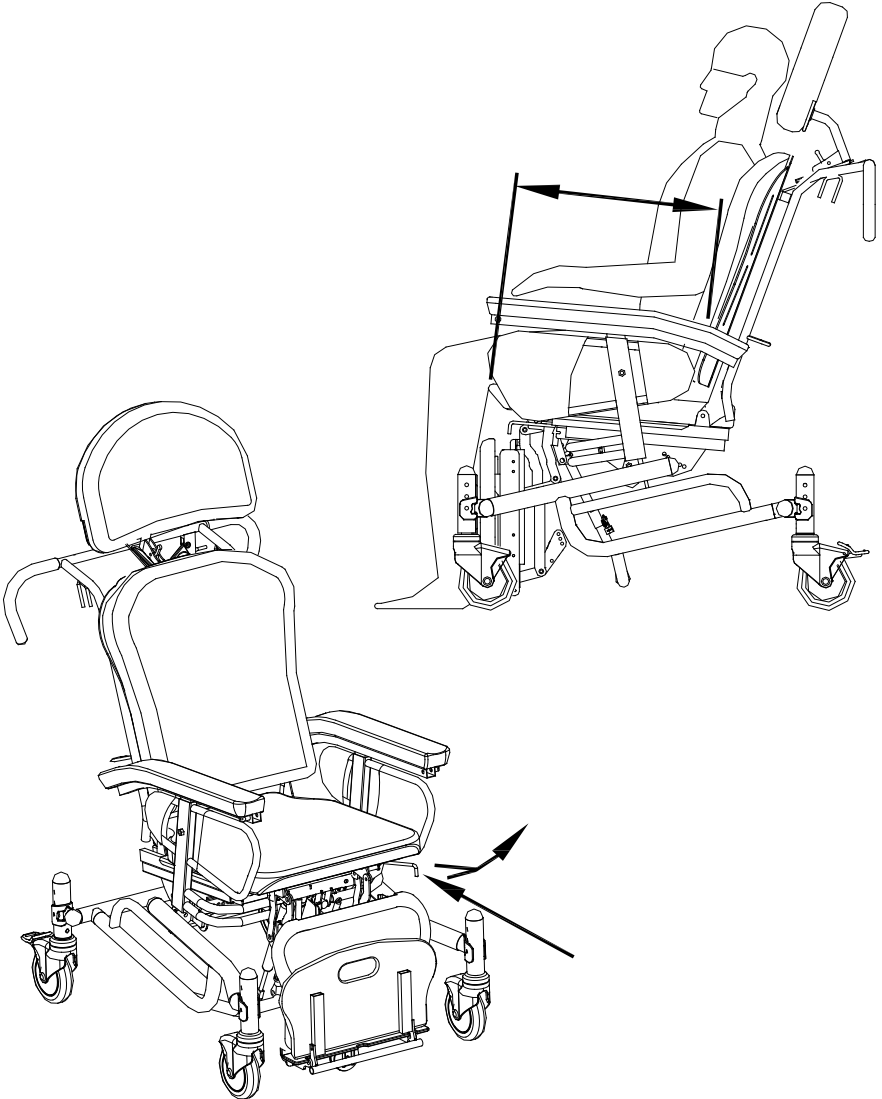- 2.1. Determine if the seat depth requires adjustment.
- 2.2. To adjust the seat depth, grip the handle behind the seat at front left, push it towards the left side and slide the seat to the correct position.
- 2.3. Check if the seat is correctly adjusted for the occupant, correct if required.

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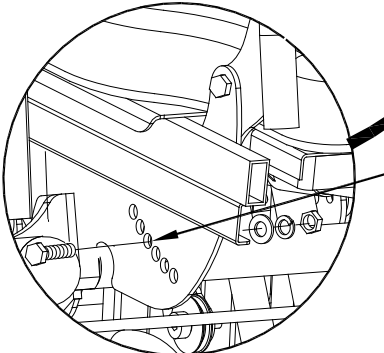
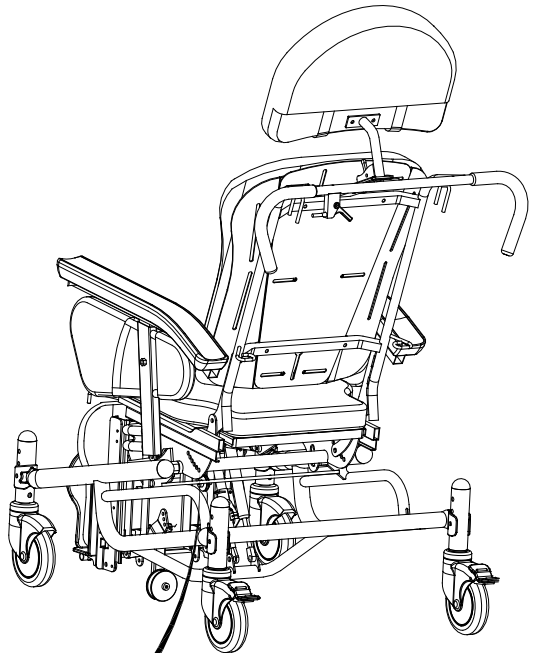
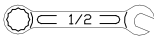
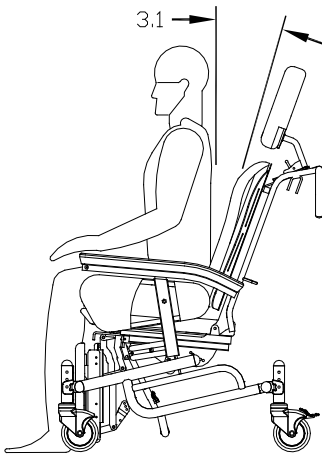
3. Seat/back angle adjustment.

- 3.1. When the chair is unoccupied, determine if the seat/back angle is positioned correctly.
- 3.2. Remove the 2 bolts (1 per side) behind the chair, choose the correct angle.
- 3.3. Set the backrest to the correct angle, replace and tighten all bolts.


BE SURE EACH SIDE IS AT THE SAME LEVEL.

- 3.4. Check if the backrest angle is correctly adjusted for the occupant, correct if required.

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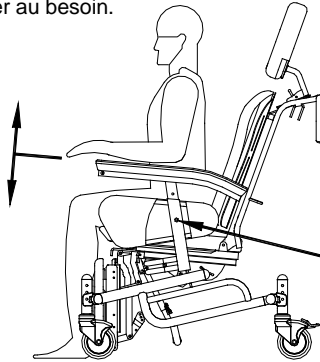
4. Armrest height.

- 4.1. Determine if the armrest height needs adjustment.
- 4.2. Remove the bolt with plastic cap on the armrest side and set the desired height. Replace and tighten securely the bolt.
- 4.3. BE SURE EACH SIDE IS AT THE SAME LEVEL. Check the height for proper fit and modify if necessary.

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

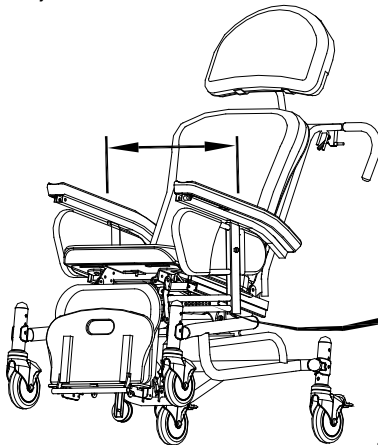
5. Armrest width.

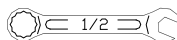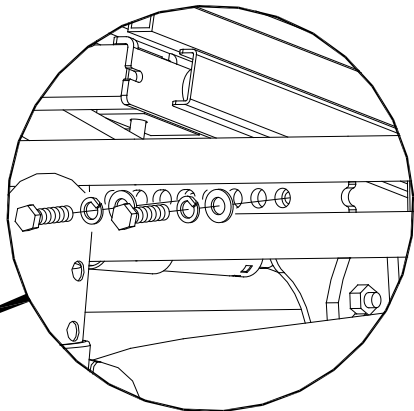- 5.1. When the chair is unoccupied, determine if the armrest width needs adjustment.
- 5.2. Remove the 2 bolts at the armrest side. Move the armrest left or right to the desired width. Replace the 2 bolts and the lock washers and securely tighten.
- 5.3. Check the seat is correctly adjusted for the occupant, correct if required.

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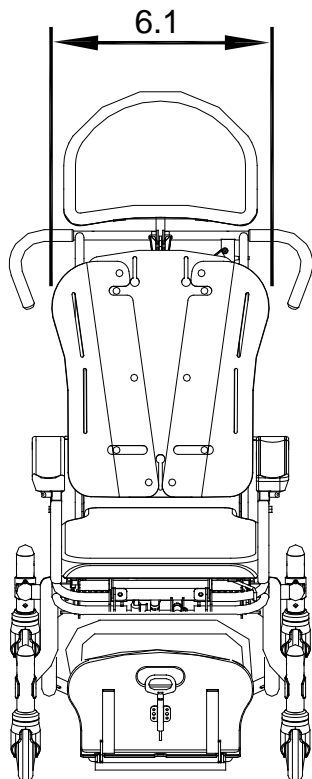
FRONT VIEW
VUE DE FACE

6. Backrest width.

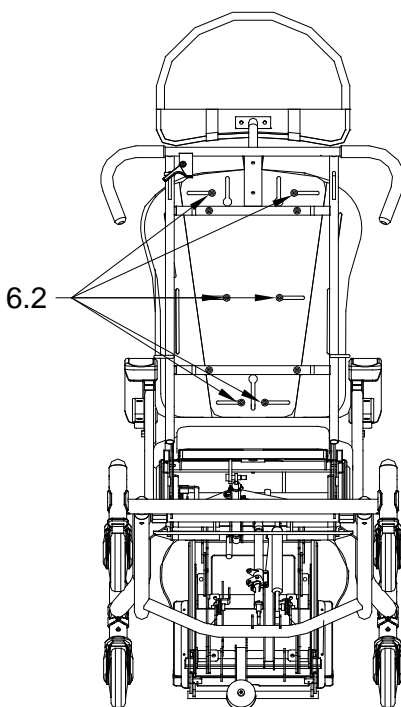
- 6.1. When the chair is unoccupied, determine if the backrest width needs adjustment.
- 6.2. Loosen the 6 screws on the back of the backrest and slide the plate to the desired width.
- 6.3. BE SURE EACH SIDE IS AT THE SAME WIDTH. Check the backrest width is correctly adjusted for the occupant, correct if required.

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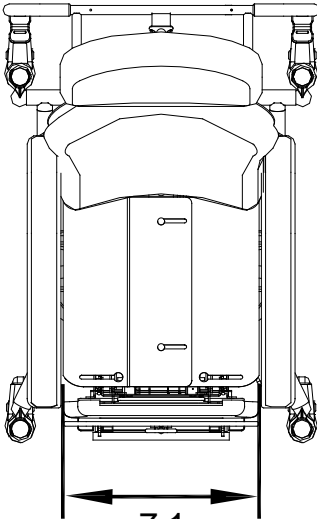
TOP VIEW
VUE DE DESSUS

7. Seat width.

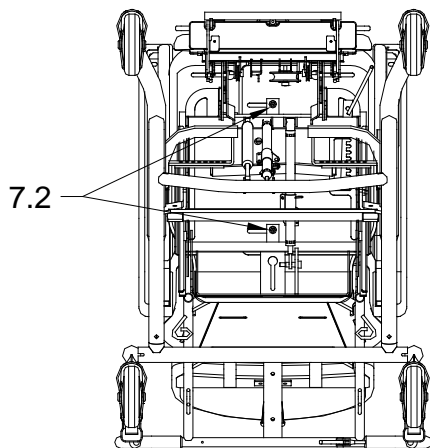- 7.1. First, determine if the seat width needs adjustment.
- 7.2. When the chair is unoccupied, loosen the screws under the seat and slide the plates to the right width.
- 7.3. BE SURE EACH SIDE IS AT THE SAME WIDTH. Retighten securely all bolts. Check if the chair is correctly adjusted for the occupant, correct if required.

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



8. Footrest height.

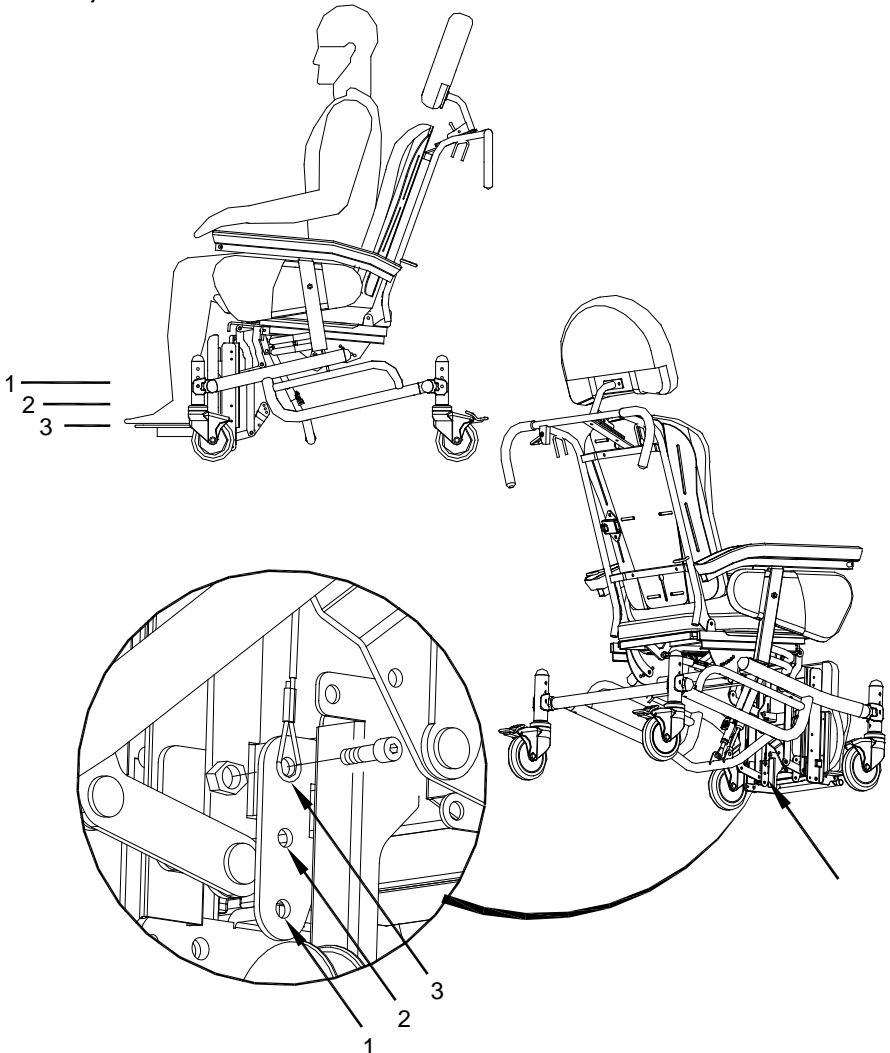- 8.1. When the chair is unoccupied, determine if the footrest height needs adjustment.
- 8.2. On each side of the footrest remove the insert cable bolt. BE SURE EACH SIDE IS AT THE SAME LEVEL AND REPLACE THE BOLTS AND LOCK WASHERS. Tighten securely all bolts.
- 8.3. Check if the chair is correctly adjusted for the occupant, correct if required.

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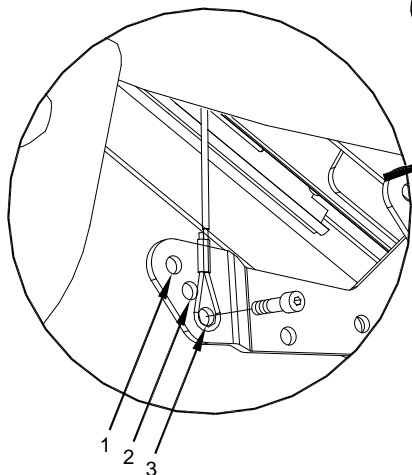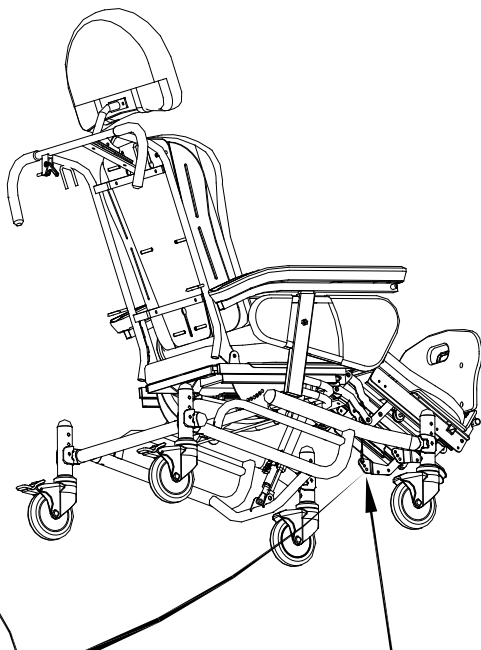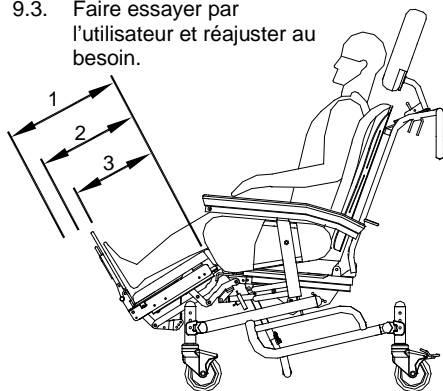
9. Footrest extension.

- 9.1. First, determine if the footrest length needs adjustment. Pull the handle below the footrest to correct the length.
- 9.2. Remove the bolt at each side of the footrest, extend it and replace the bolt. BE SURE EACH SIDE IS AT THE SAME LEVEL. Tighten securely all bolts.
- 9.3. Check if the chair is correctly adjusted for the occupant, correct if required.

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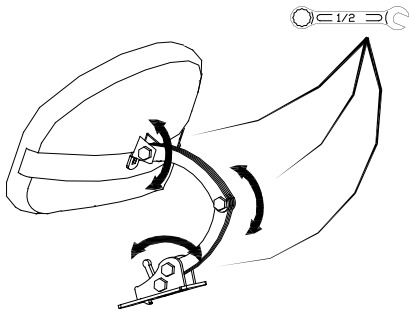
10. Adjustable Headrest.

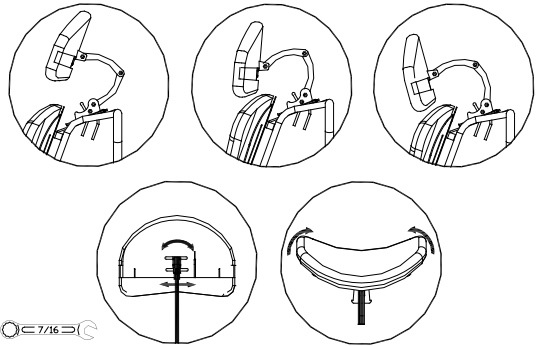- 10.1. Determine the desired position of the headrest.
- 10.2. Slightly loosen the adjustment bolts, DO NOT REMOVE, adjust the headrest and re-tighten the bolts.
- 10.3. The plate can be bent to fit the user's head shape.
- 10.4. Check if the chair is correctly adjusted for the occupant, correct if required. Retighten securely all bolts.

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


11. Lateral and shoulder supports height adjustment.

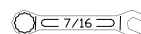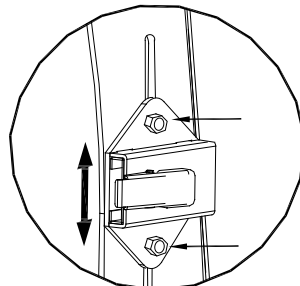- 11.1. Determine the correct position of the lateral or the shoulders supports.
- 11.2. Slightly loosen the adjustment bolts, DO NOT REMOVE.
- 11.3. Adjust each side to the same level and re-tighten securely the adjustment bolts.
- 11.4. Check if the lateral and shoulders support is correctly adjusted for the occupant, correct if required.

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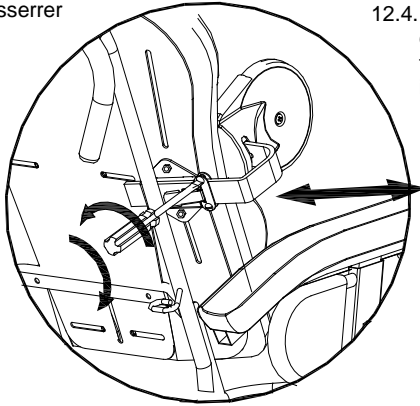
12. Lateral and shoulder supports width adjustment.

- 12.1. Determine if the lateral or shoulder supports need adjustment.
- 12.2. Slightly loosen the 2 adjustment lock screws with a screwdriver. DO NOT REMOVE.
- 12.3. Move the lateral supports to the desired position BE SURE EACH SIDE IS AT THE SAME WIDTH and retighten securely the screws.
- 12.4. Check if the chair is correctly adjusted for the occupant, correct if required.

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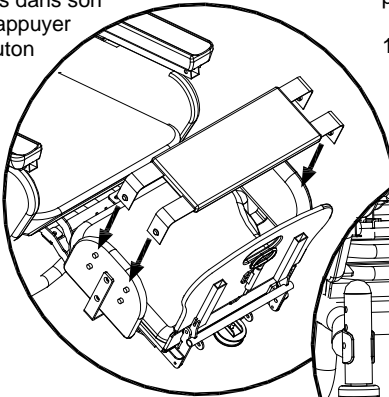
13. Lateral leg protectors.

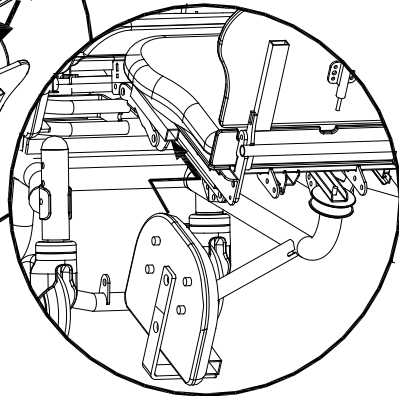
- 13.1 Press in the pressure release lock button, then slide the lateral leg protectors into its support.
- 13.2 Secure the leg protectors cushion with the pressure snaps (2 snaps on each side).

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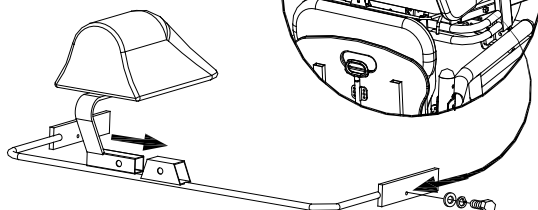14. Foldaway perineal abductor pad.

- 14.1 Slide the fold-away perineal abductor pad support in the cavity.
- 14.2 Tighten the two screws.
- 14.2 Slide the fold-away perineal abductor pad in its support and press in the release pressure lock button.

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15. 13'' wheel seat adjustment.

- 15.1. Determine if the seat height should be adjusted up or down.
- 15.2. When the chair is unoccupied, unscrew and remove the 13'' wheel's bolts. (2 bolts)
- 15.3. Determine the desired height and re-attach the structure with the 13'' wheels to the cylindrical support and tighten it using bolts, nuts and washers supplied. Securely tighten all bolts.

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

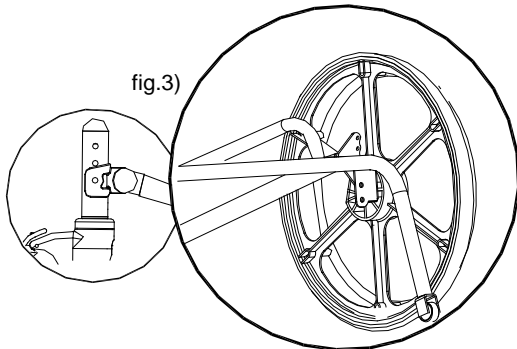
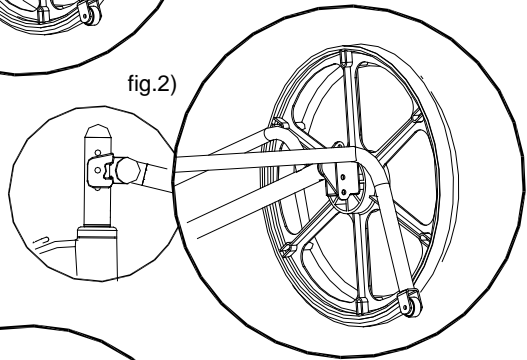
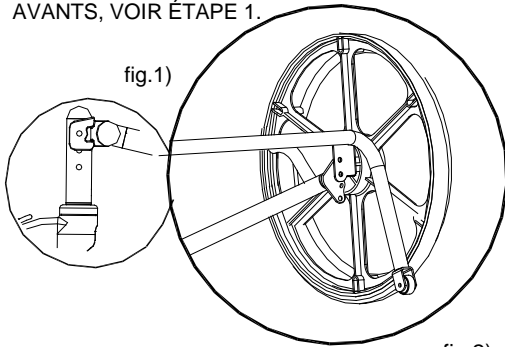
16. 19" wheel Seat height adjustment .

- 16.1. Determine if the seat height should be adjusted up or down.
- 16.2. When the chair is unoccupied, unscrew and remove the 19" wheel's (4 bolts).
- 16.3. Determine the desired height and re-attach the structure with the 19" wheels to the wheel support and tighten it using the bolts, nuts and washers provided. Securely tighten all bolts.

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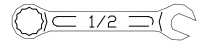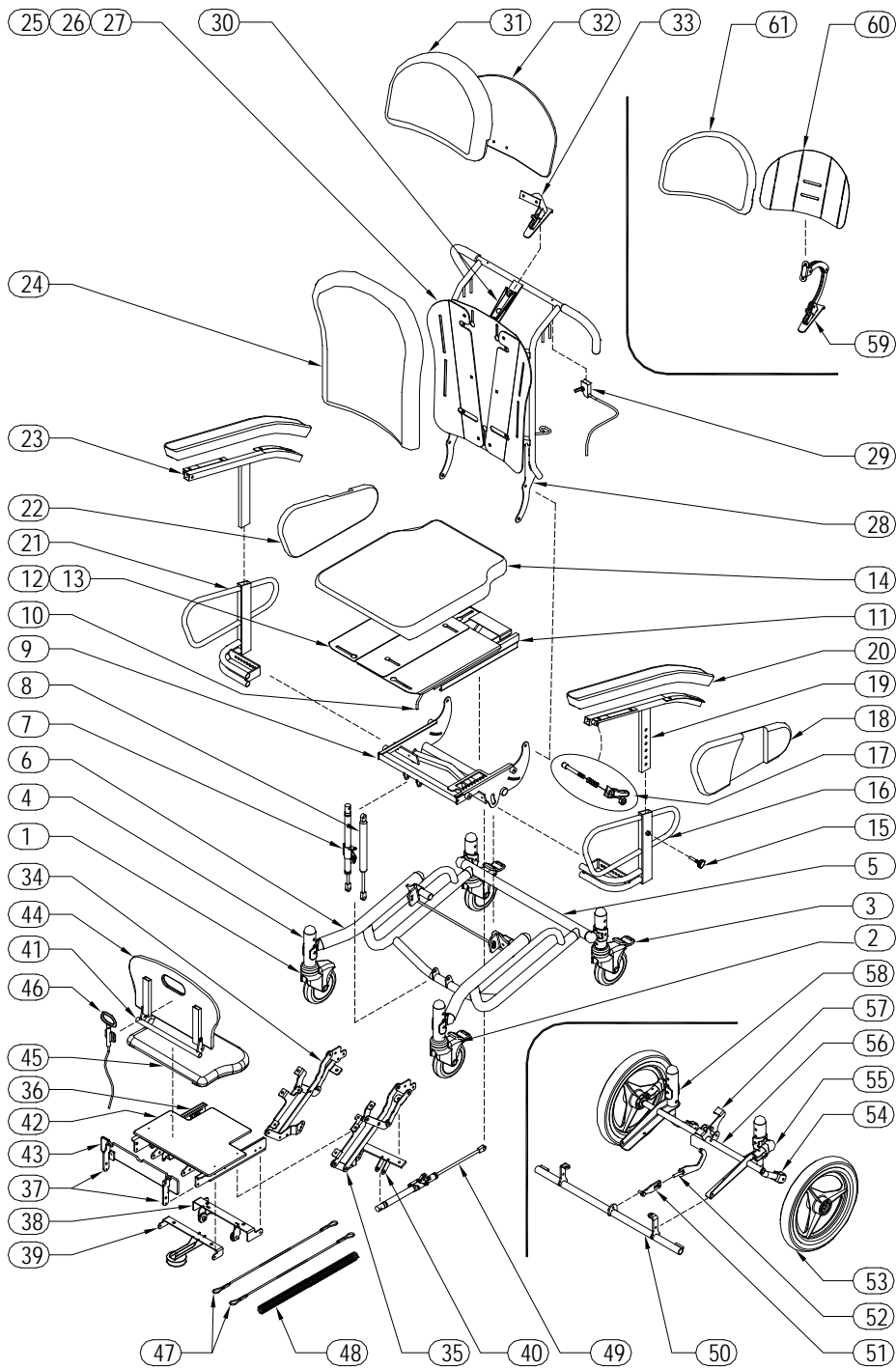Always specify fabric name, pattern, color, and the Chair's serial number for all replacement part orders. Upholstered replacement parts may vary slightly in color from the original part due to dye lot differences. LPA Medical will not be responsible for dye lot differences

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TERMS & CONDITIONS

GENERAL:

• All items & prices listed are subject to change without notice. We reserve the right to change and/or improve any product as deemed advisable without notification. Shipments will be made at prevailing prices at time of shipment.

FREIGHT CLAIMS:

• LPA MEDICAL can not accept claims for merchandise that has been damaged in transit. Examine all merchandise upon delivery. If damage has occurred, it is the consignee's responsibility to file a claim with the carrier immediately if damage is apparent upon delivery or within 5 days if it is subsidized damage.

RETURN GOODS

• Merchandise to be returned must have prior written authorisation from our Customer Service Department. All request most include the following information:

Reason for requested return

LPA MEDICAL invoice or acknowledgement number

LPA MEDICAL model or part number and number of cartons and items being returned.

Unauthorised returns will not be accepted. All authorised returns must be shipped pre-paid except where there has been an error in processing or a defect in materials or workmanship, in which case Customer Service will issue special instructions.

WARRANTY

• LPA MEDICAL warranty covers all defects in workmanship, except the exclusions listed in the not covered section below, for the following periods of time from date of purchase:

Lifetime for metal parts (frame): R-505, R-605, R-705, T-40, U-110, U-120, W-2, Z-15, Z-25 chairs

5 years for metal parts (frame): C-207, Optimum, O-07, R-505/605/705-SHD, S-150, S-250, Z-45 chairs

3 years only for auto lock glider mechanism

1 year for all other mechanism, gas spring cylinder or mechanical lock, control cable, wheels & casters, tray, upholstery.

If service is required under this warranty, contact our Customer Service Department in writing, giving the following information

- 1- Model or part number involved
- 2- Date of purchase, including LPA MEDICAL invoice or acknowledgement number
- 3- A description of the problem.

WHAT LPA MEDICAL WILL COVER

• LPA MEDICAL will at our option repair or replace at no charge to you, the original purchaser, the defective product or parts during the periods of given time

WHAT LPA MEDICAL WILL NOT COVER

- Normal wear and tears (frame, wheels, fabrics)
- Accidental damage including fire and water
- Fabric fading and metal oxidation (rust)
- Cost of transportation to or from LPA MEDICAL factory
- Unauthorised repairs made or attempted
- Damage resulting from abnormal or unreasonable use.

Your Rights Under Province or state law. This warranty gives you specific legal rights, you may also have other rights which vary from Province to Province or State to State.

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SPECIAL NOTES ABOUT WHEELCHAIR TIE-DOWN WHILE TRANSPORTED

LPA Medical recommends that wheelchair users NOT be transported in vehicles of any kind while seated in wheelchairs.

As of this date, the Department of Transportation has not approved any Tie down systems for transportation of a user while in a wheelchair, in a moving vehicle of any type.

It is **LPA Medical's** position that users of wheelchairs should be transferred into appropriate seating in vehicles for transportation and use be made of the restraints made available by the automobile industry.

LPA Medical cannot and does not recommend any wheelchair transportation systems.

REGARDING RESTRAINTS - SEAT POSITIONING STRAPS - IT IS THE OBLIGATION OF THE DEALER, THERAPIST, TRANSPORTER AND OTHER HEALTH CARE PROFESSIONAL IN CHARGE TO DETERMINE IF A SEAT POSITIONING STRAP IS REQUIRED TO ENSURE THE SAFE OPERATION OF THIS EQUIPMENT BY THE USER. SERIOUS INJURY CAN OCCUR IN THE EVENT OF A FALL FROM A WHEELCHAIR.

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Adresse

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FOR YOUR RECORDS

Purchaser's record for LPA MEDICAL WARRANTY

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