# ***DYN-ERGO***

Modèles S-150-16, S-150-18,  
S-150-20, S-150-22, S-150-24

Models S-150-16, S-150-18,  
S-150-20, S-150-22, S-150-24

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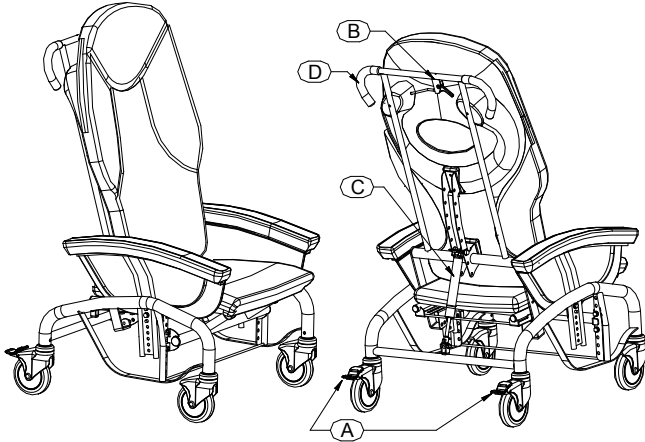
## ***User's Guide***

*Your DYN-ERGO Scoot Chair is built for years of trouble-free service. Thoroughly read and follow the instructions to properly adjust, operate and care for your chair.*

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## Dyn-Ergo Scoot Chair Usage

For safest operation, please familiarize yourself with the following components:

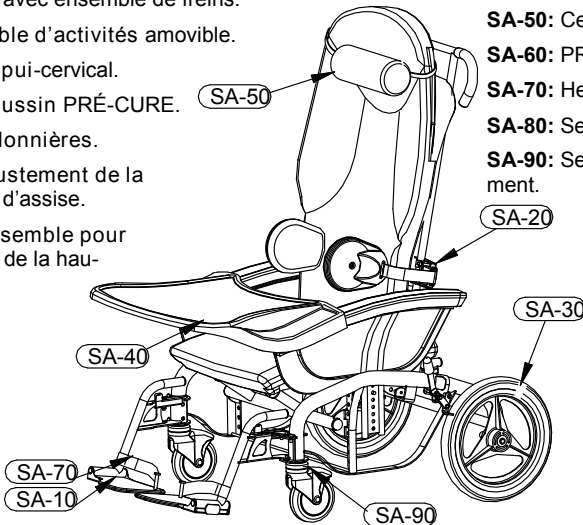
- A. Brake on each rear wheel.
- B. Tilt-In-Space lever activator.
- C. Gas spring.
- D. Push handle.

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## Available Options

- SA-10**: Removable Swing-Away Footrests.
- SA-20**: Removable structural Lateral Supports adjust to provide comfortable correct trunk positioning.
- SA-30**: 13" Rear wheels with brakes.
- SA-40**: Removable activity tray.
- SA-50**: Cervical headrest.
- SA-60**: PRE-CURE cushion.
- SA-70**: Heel support loops.
- SA-80**: Seat depth adjustment.
- SA-90**: Seat height adjustment.

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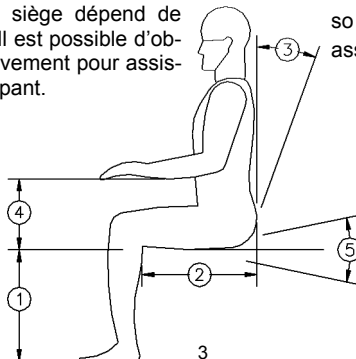
## WARNING

All brakes **MUST BE ENGAGED AT ALL TIMES** except during transportation.

Be sure all adjustments are correct and bolts and nuts are tightened before sitting patient in the chair.

All adjustments must be supervised by a healthcare professional.

This product is intended for use in a secure environment and under constant supervision.

### Dyn-Ergo chair's maintenance

For best performance, it is necessary that the chair be kept clean and all parts adjusted properly.

When needed, the upholstered surfaces must be washed with a mild detergent diluted or dissolved in water. If necessary, a mild clear disinfectant can be used for hygienic purposes.

Metal parts of the chair should also be kept clean using the same mild detergent solution as above.

### Available adjustments

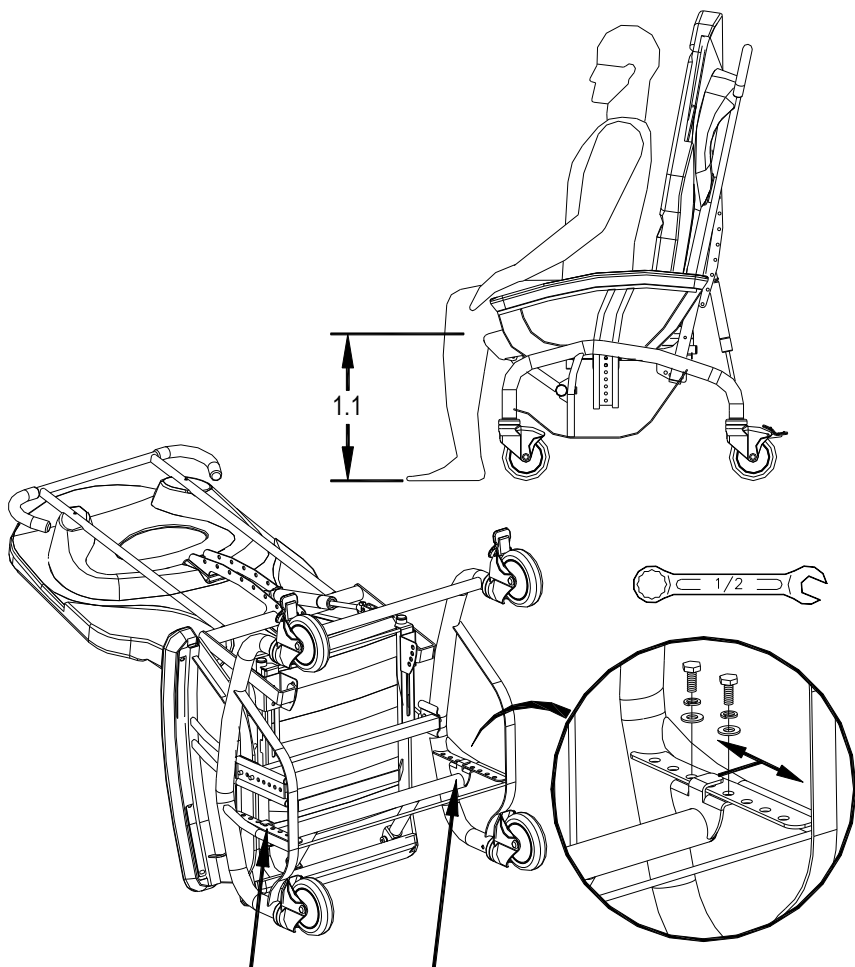
1. The seat height from floor is adjustable from 13 ½" to 18 ½" by 1" increments.
2. Seat depth adjustable from 13 ¾" to 18".
3. Angle between seat and backrest is adjustable from 95° to 119° by 8° increments.
4. The armrest height from the seat is adjustable from 8" to 9 ½" by ¾" increments.
5. A safer egress can be obtained by modifying the tilt angle gas spring so that the chair provides a lift assist when exiting the chair.

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

## 1. Seat height adjustments

- 1.1. Determine if the seat height should be adjusted up or down.
- 1.2. Tip the unoccupied chair forward. Un-screw and remove the 4 bolts on the seat height adjustment pivot structure as shown below.
- 1.3. Move the seat height adjustment pivot structure to the desired position. MAKE SURE BOTH SIDES ARE LEVEL, tighten the 4 bolts securely with the lock washers in place.
- 1.4. Put the chair in an upright position and check that the seat is at the proper height for the occupant. Readjust as required.

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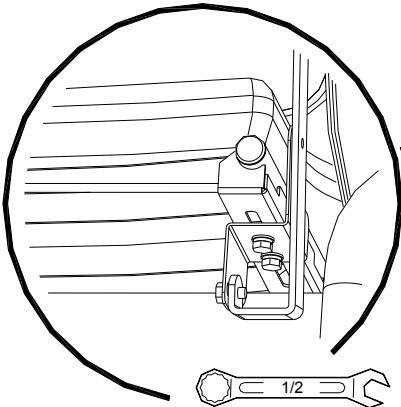
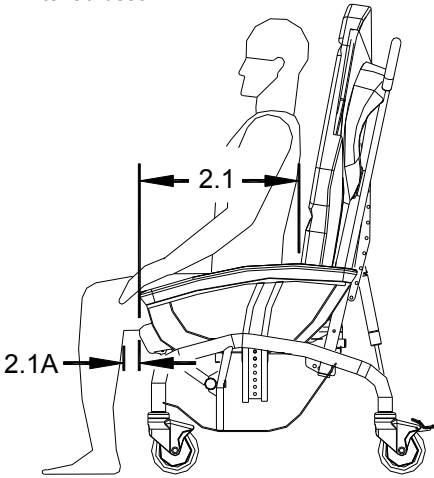
## 2. Seat depth adjustments

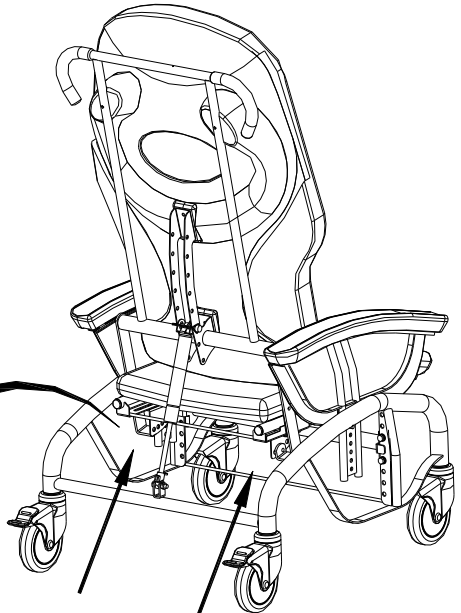- 2.1. Determine if the seat depth is adjusted to allow proper clearance behind the user's knees. (2.1A). Seat depth is adjusted by moving the backrest.
- 2.2. When the chair is unoccupied, behind the chair, just below the seat, loosen the 4 bolts through the seat slot. DO NOT REMOVE THESE BOLTS; just loosen them enough to be able to move the backrest forward or backward over the seat deck. Slide both sides of the backrest simultaneously until the desired depth is achieved.
- 2.3. When the seat is at the proper depth, re-tighten the 4 bolts.
- 2.4. Check that the depth is adapted for the occupant. Repeat the adjustment process as needed.

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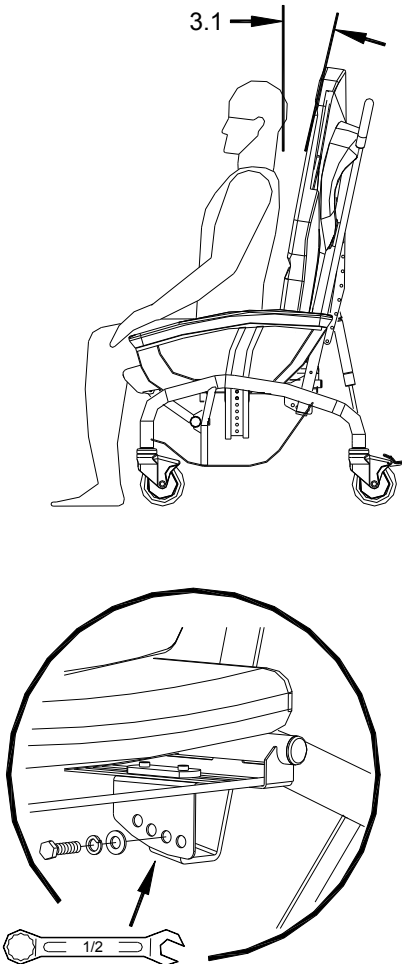
### 3. Seat/back angle adjustments

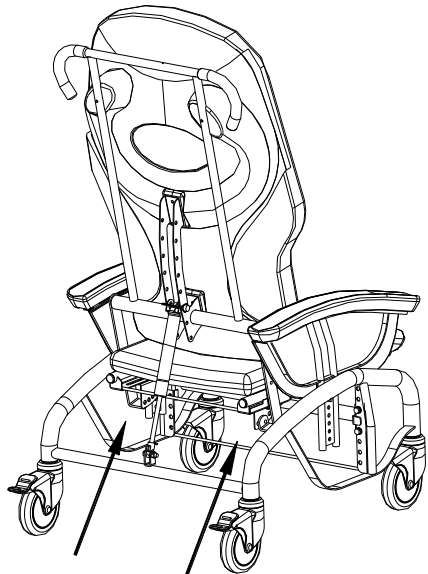- 3.1. Determine if the seat/back angle is positioned correctly.
- 3.2. When the chair is unoccupied, behind the chair, just below the seat deck, the backrest is attached along an arc with 4 holes. Remove the bolts from both sides of the arc.
- 3.3. Push or pull the top of the backrest to modify the seat/back angle. When the desired angle is achieved, align the attachment holes and insert the bolts with lock washers in place. Tighten securely. MAKE CERTAIN THAT THE BACKREST IS ATTACHED AT THE SAME HOLE ON EACH SIDE.
- 3.4. Check the angle for proper fit and readjust as needed.

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



## 4. Armrest height adjustments

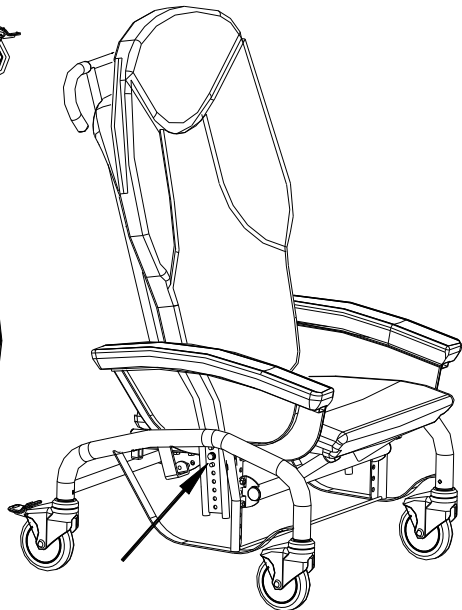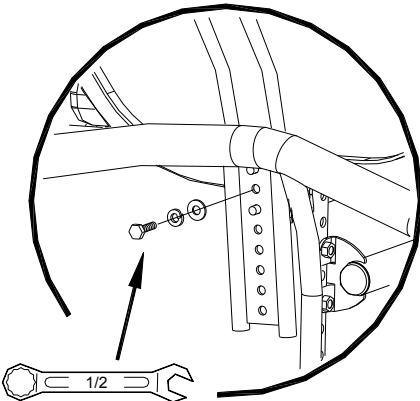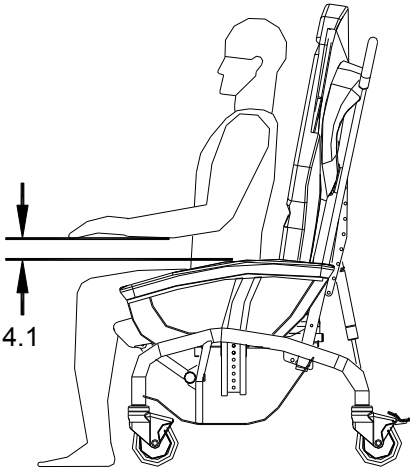- 4.1. Determine if the armrest height needs adjustment.
- 4.2. Remove the bolt on the side of the armrest. Move the armrest up or down to the desired height. Replace the bolt with its flat washer and lock washer.
- 4.3. In most cases, it is best to adjust both armrests to the same height.

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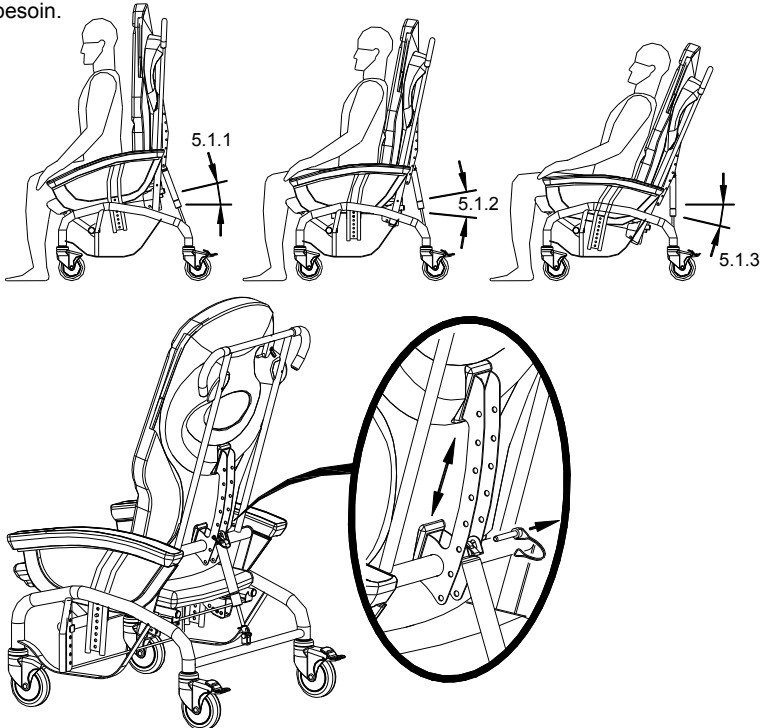## 5. Tilt angle adjustments

- 5.1. The seat tilt angle can be adjusted to accommodate the needs of the user by changing the gas spring attachment location on the backrest bracket. Tilt angle ranges include:
  - 5.1.1. A high lift-assist angle to aid egress
  - 5.1.2. An all-around medium angle position
  - 5.1.3. An inclined tilt angle to prevent slumping.
- 5.2. Determine the best tilt range for the user. With the chair unoccupied, remove the locking pin from the gas spring. Move the gas spring end along the attachment bracket until the desired tilt angle is achieved. Reattach the gas spring with the locking pin and secure the pin with the safety loop. Test the chair by actuating the hand lever and moving the chair through the entire tilt range to see if the desired angle is achieved. **MAKE CERTAIN THAT THE CABLE IS NOT PINCHED.**
- 5.3. Readjust as required.

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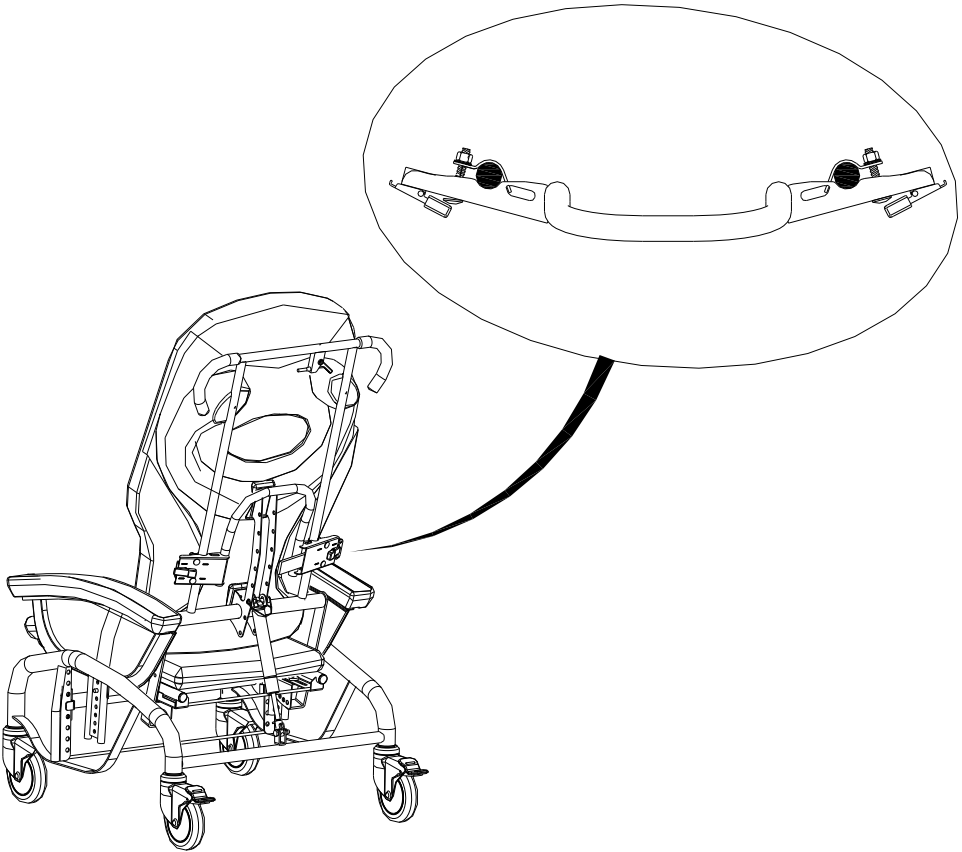
## 6. Lateral supports height

- 6.1. Determine the lateral supports needed height.
- 6.2. Loosen the adjustment bolts. **DO NOT REMOVE THEM.**
- 6.3. Set the lateral supports to the desired height and tighten the bolts.
- 6.4. Check that the lateral supports are adapted to the occupant, readjust as required.

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

## 7. Lateral support width adjustments

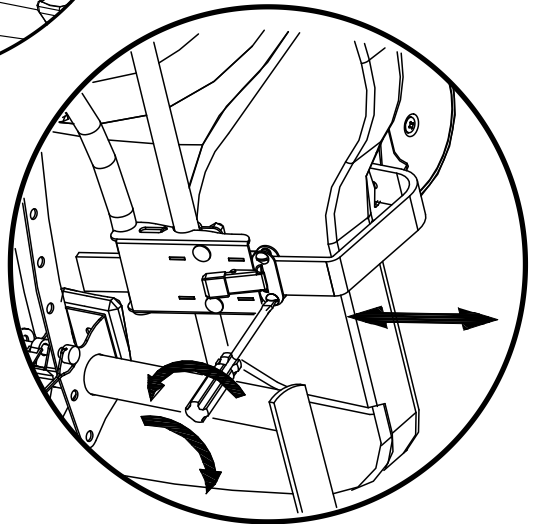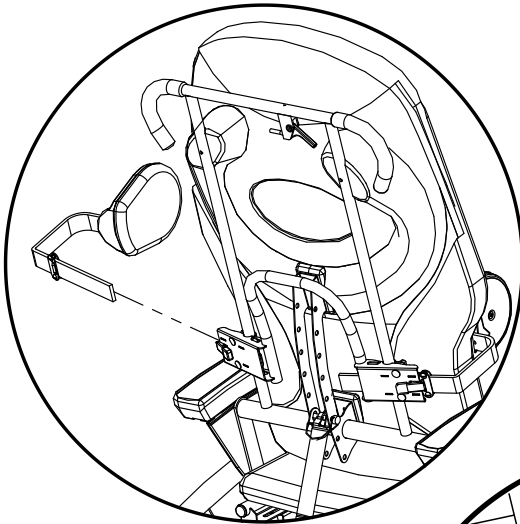- 7.1. Determine if the lateral supports need adjustment.
- 7.2. Loosen the two lock adjustment screws with a screwdriver. DO NOT REMOVE THE SCREWS.
- 7.3. Move the lateral supports to the desired position. BE SURE EACH SIDE IS AT THE SAME WIDTH and tighten the screws.
- 7.4. Check that the lateral support fit the occupant, readjust as required.

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

## 8. Footrest positioning

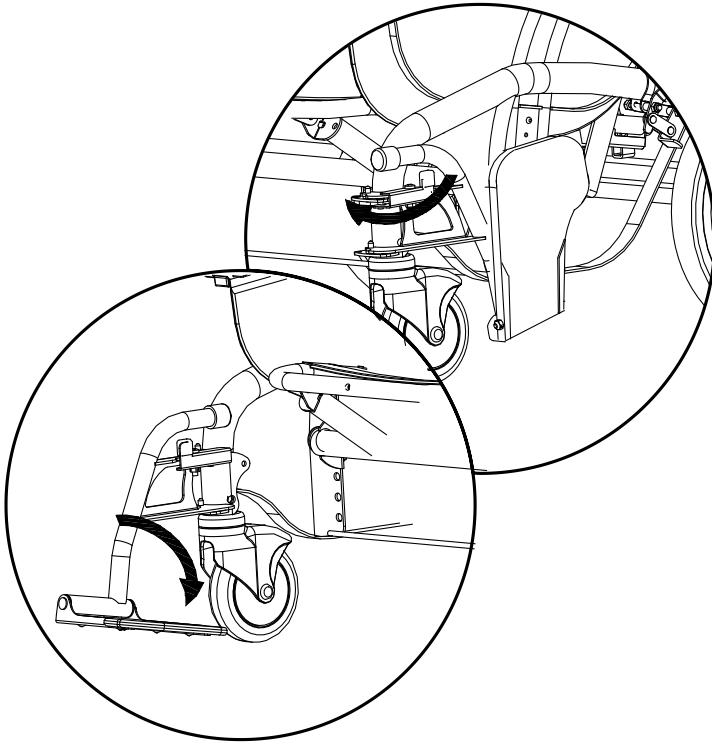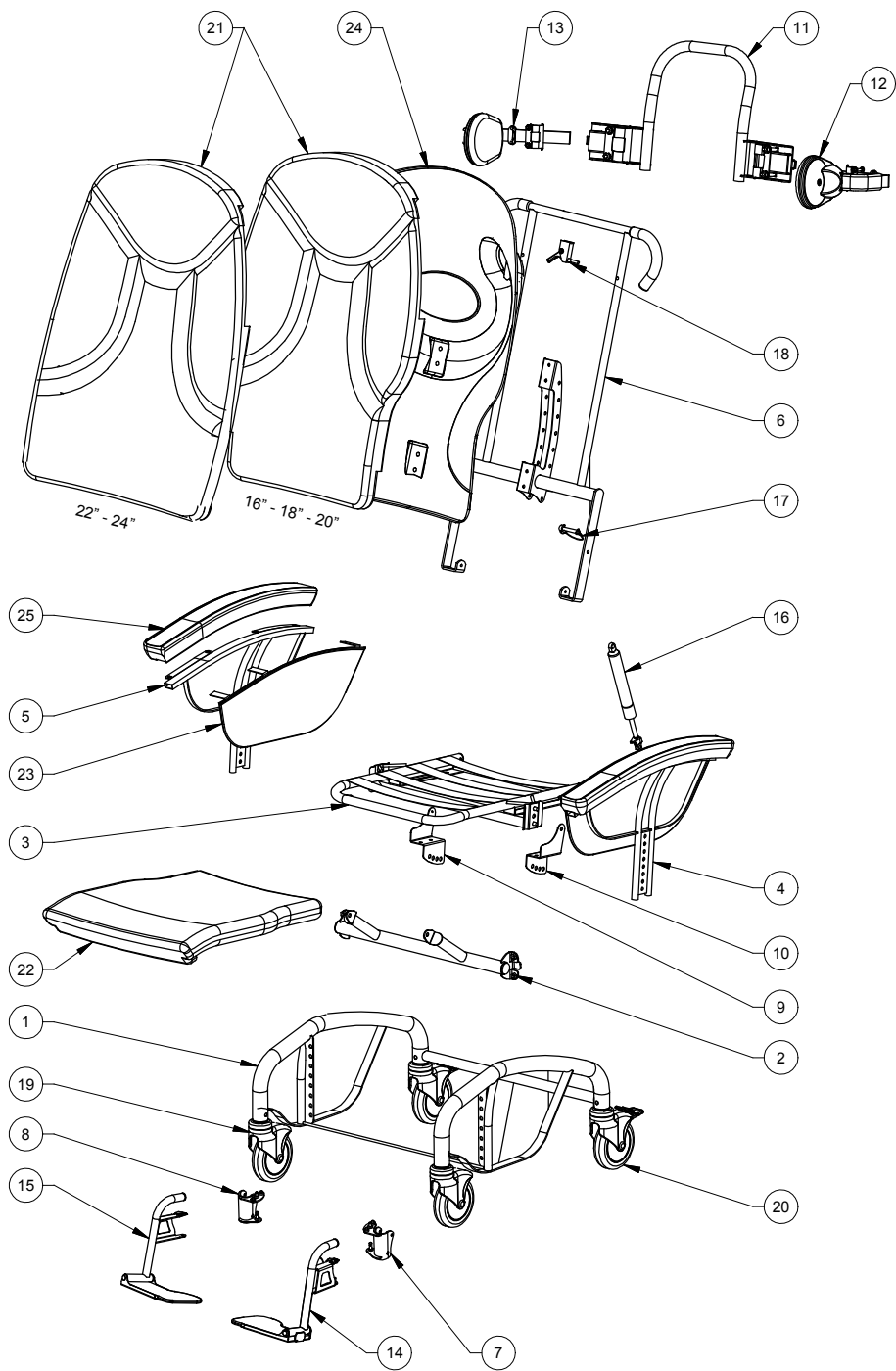- 8.1. Slightly raise the footrest structure.
- 8.2. Rotate the footrest structure by 180 degrees.
- 8.3. Lower the footrest paddle.

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### **Note "A"**

Replace "xx" in part N° with chair width (16", 18", 20", 22" or 24"). For cushions, use 18" for a 20" chair and 24" for a 22" chair.

### **Important : Note "C"**

Always specify fabric name, pattern, color, and the Dyn-Ergo's serial number for all replacement part orders. Upholstered replacement parts could vary slightly in color from the original part due to dye lot differences. LPA Medical will not be responsible for dye lot differences.

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## TERMS & CONDITIONS

### GENERAL:

- All items & prices listed are subject to change without notice. We reserve the right to change and/or improve any product as deemed advisable without notification. Shipments will be made at prevailing prices at time of shipment.

### FREIGHT CLAIMS:

- LPA MEDICAL cannot accept claims for merchandise that has been damaged in transit. Examine all merchandise upon delivery. If damage has occurred, it is the consignee's responsibility to file a claim with the carrier immediately if damage is apparent upon delivery or within 5 days if it is subsided damage.

### RETURN GOODS

- Merchandise to be returned must have prior written authorisation from our Customer Service Department. All request must include the following information:  
Reason for requested return  
LPA MEDICAL invoice or acknowledgement number  
LPA MEDICAL model or part number and number of cartons and items being returned.  
Unauthorised returns will not be accepted. All authorised returns must be shipped pre-paid except where there has been an error in processing or a defect in materials or workmanship, in which case Customer Service will issue special instructions.

### WARRANTY

- LPA MEDICAL warranty covers all defects in workmanship, except the exclusions listed in the not covered section below, for the following periods of time from date of purchase:  
Lifetime for metal parts (frame): R-505, R-605, R-705, T-40, U-110, U-120, W-2, Z-15, Z-25 chairs  
5 years for metal parts (frame): C-207, Optimum, O-07, R-505/605/705-SHD, S-150, S-250, Z-45 chairs  
3 years only for auto lock glider mechanism  
1 year for all other mechanism, gas spring cylinder or mechanical lock, control cable, wheels & casters, tray, upholstery.  
If service is required under this warranty, contact our Customer Service Department in writing, giving the following information  
1. Model or part number involved  
2. Date of purchase, including LPA MEDICAL invoice or acknowledgement number  
3. A description of the problem.

### WHAT LPA MEDICAL WILL COVER

LPA MEDICAL will at our option repair or replace at no charge to you, the original purchaser, the defective product or parts during the periods of given time

### WHAT LPA MEDICAL WILL NOT COVER

- Normal wear and tears (frame, wheels, fabrics)
- Accidental damage including fire and water
- Fabric fading and metal oxidation (rust)
- Cost of transportation to or from LPA MEDICAL factory
- Unauthorised repairs made or attempted
- Damage resulting from abnormal or unreasonable use.

Your Rights Under Province or state law. This warranty gives you specific legal rights, you may also have other rights which vary from Province to Province or State to State.

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**SPECIAL NOTES ABOUT WHEELCHAIR TIE-DOWN WHILE TRANSPORTED**

**LPA Medical Inc.** recommends that wheelchair users NOT be transported in vehicles of any kind while seated in wheelchairs.

As of this date, the Department of Transportation has not approved any Tie down systems for transportation of a user while in a wheelchair, in a moving vehicle of any type.

It is **LPA Medical's** position that users of wheelchairs should be transferred into appropriate seating in vehicles for transportation and use be made of the restraints made available by the automobile industry.

**LPA Medical** cannot and does not recommend any wheelchair transportation systems.

**REGARDING RESTRAINTS - SEAT POSITIONING STRAPS - IT IS THE OBLIGATION OF THE DEALER, THERAPIST, TRANSPORTER AND OTHER HEALTH CARE PROFESSIONAL IN CHARGE TO DETERMINE IF A SEAT POSITIONING STRAP IS REQUIRED TO ENSURE THE SAFE OPERATION OF THIS EQUIPMENT BY THE USER. SERIOUS INJURY CAN OCCUR IN THE EVENT OF A FALL FROM A WHEELCHAIR.**

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IN THE EVENT OF A DISCREPANCY BETWEEN THIS USER'S GUIDE AND OUR WEB SITE, THE LATTER SHALL PREVAIL.

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**Purchaser's record for LPA MEDICAL WARRANTY**

Date d'achat :

Date of purchase

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Purchased from

Adresse :

Address

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Model No., serial No. and quantity

# DYN-ERGO

Modèles S-150-16, S-150-18,  
S-150-20, S-150-22, S-150-24

Models S-150-16, S-150-18,  
S-150-20, S-150-22, S-150-24

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Update: March 11<sup>th</sup>, 2016

**LPA MÉDICAL INC.**

**2527 Avenue Dalton, Québec, (Qc), Canada, G1P 3S6**

Tel : (418) 681-1313, 1-800-663-4863 Fax : (418) 681-4488

[lpa@lpamedical.com](mailto:lpa@lpamedical.com)

<http://www.lpamedical.com>

**OPTIMA PRODUCTS INC.**

**2230 Edgewood Avenue – STE 3A,  
St. Louis Park, MN 55426 USA**

Tel: (952) 745-0027

Toll free: 1-888-990-9394

Fax: (952) 745-0197

[optima@optimaol.com](mailto:optima@optimaol.com)

<http://www.optimaol.com>