Nomade II

Mobile Comfort Chair

Z-45 Model
with options

Fauteuil de repos mobile

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User's Guide

Your NOMAD chair is built for years of comfort and trouble free service. Thoroughly read and follow the instructions step by step to find out how to adjust, operate and care for your NOMAD chair.

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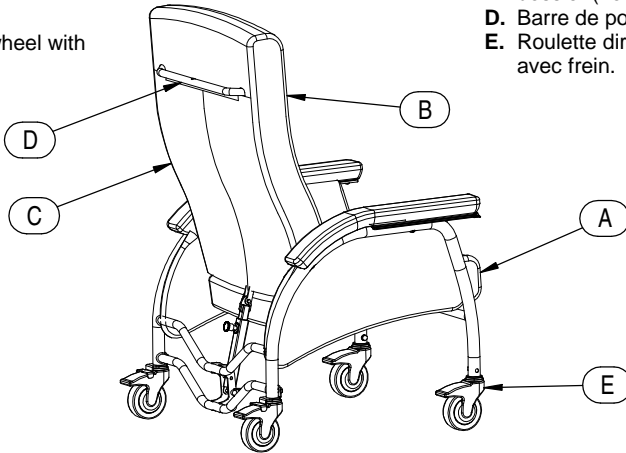
Nomad II Chair Description

For safest operation, please familiarize yourself with the following components:

- A. Seat cushion.
- B. Backrest cushion.
- C. Backrest elastic bands (hidden).
- D. Push bar.
- E. Directional wheel with brake.

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WARNING

All brakes **MUST BE ENGAGED AT ALL TIME** except during transportation.

Be sure all adjustments are correct and bolts and nuts are tightened before sitting a patient in the chair.

All adjustments must be supervised by a healthcare professional.

This product is intended for use in a secure environment under constant supervision.

Nomad II chair maintenance

To ensure the chair's optimal performance it is necessary to keep your chair clean and the parts properly adjusted.

The vinyl covered surfaces must be washed with a mild detergent diluted or dissolved in water when needed. If necessary a mild clear disinfectant may be used for hygienic purposes.

The chair's metal parts must be cleaned as well when needed using the same solution as for upholstery.

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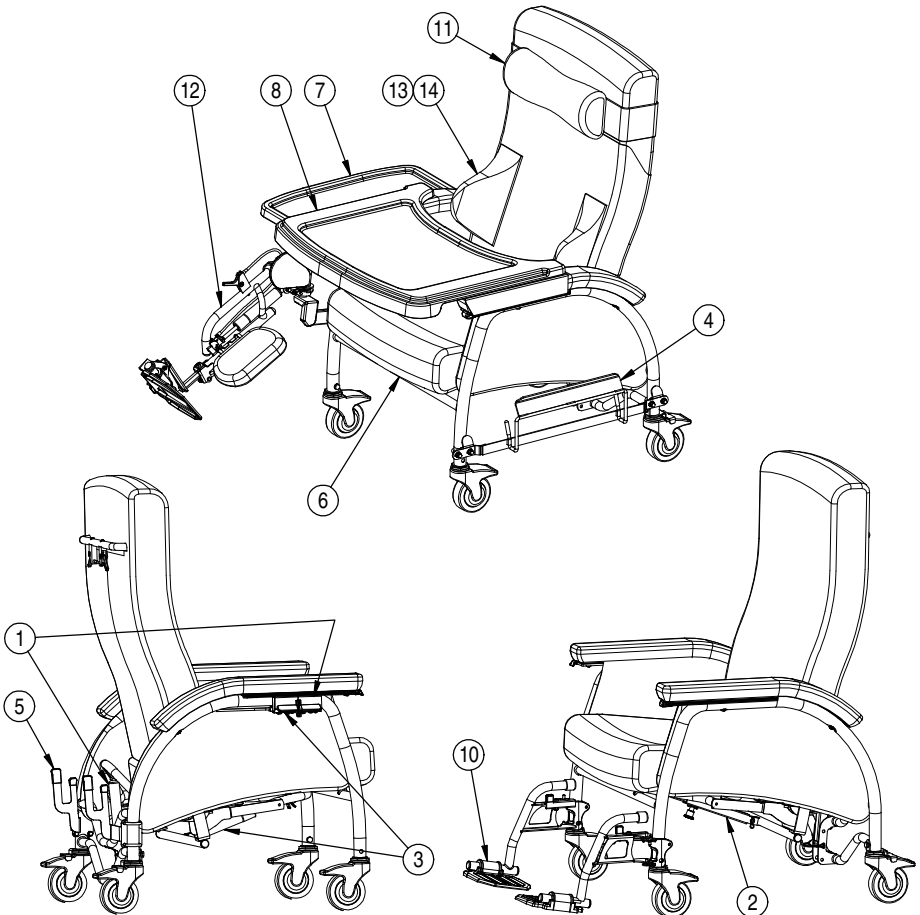
Options and accessories

1. **Z-C2-A** Gas spring operated tilt-in-space.
2. **Z-C3-A** Adjust. seat depth and backrest recline
3. **Z-C4-A** Adjust. seat depth and gas spring operated backrest recline.
4. **Z-C6** Side support bracket for storing activity tray
5. **Z-C6A** Rear support bracket for storing activity tray
6. **Z-C8** Pre-cure cushion for pressure management
7. **Z-C10** Folding lateral table
8. **Z-C14** Opaque activity tray
9. **Z-C15** Activity tray with window
10. **Z-C18** Removable foot rests (pair)
11. **Z-C20** Contoured cervical support
12. **Z-C130L/R** Adjustable elevating leg rest (left or right)
13. **RA-23** 4" Lateral trunk support
14. **RA-24** 6" Lateral trunk support

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Check out our website for a complete and up-to-date list of available options and accessories.

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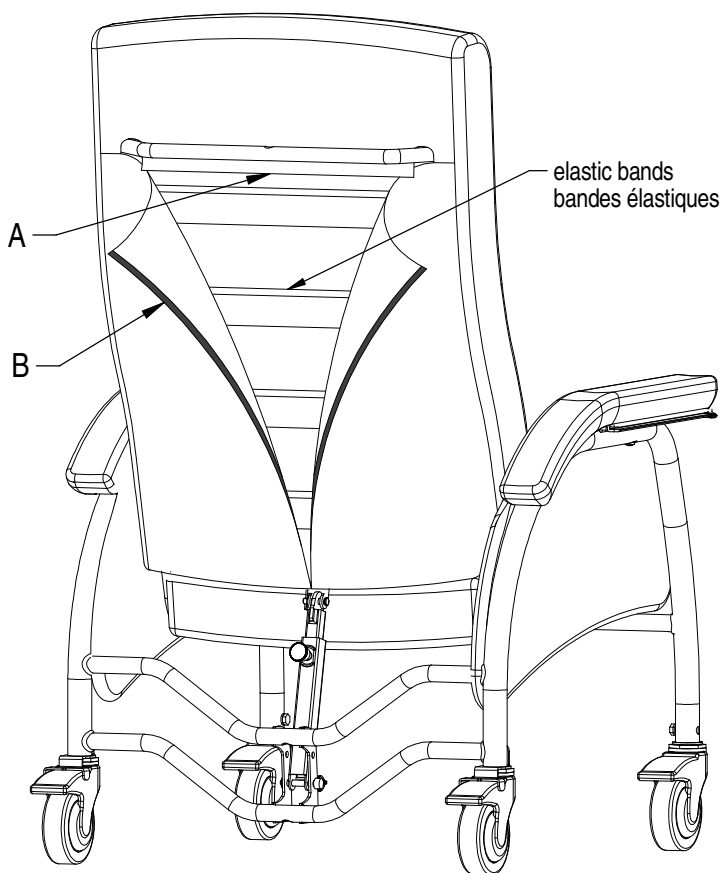
1. Backrest elastic bands adjustment

- 1.1. Detach the Velcro flap under the push bar (A).
- 1.2. Open the zipper on the rear of the backrest cushion from top to bottom (B) and push aside the flaps.
- 1.3. Place the elastic bands as desired then close the zipper and reattach the Velcro flap.

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

2. Backrest Cushion Removal

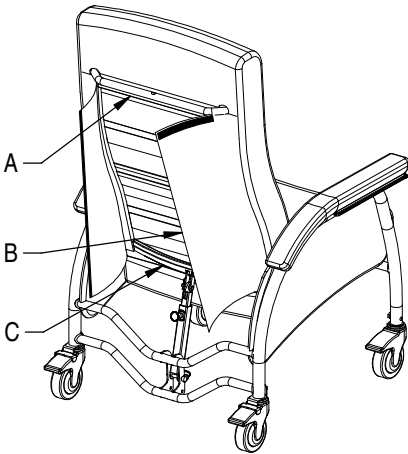- 2.1. Detach the Velcro flap under the push bar (A).
- 2.2. Open the zipper on the rear of the backrest cushion from top to bottom (B) and push aside the flaps.
- 2.3. Unhook the two black plastic J-welt from the backrest bottom steel slat and push them upfront.
- 2.4. Pull up the backrest cushion by holding its upper sides.

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
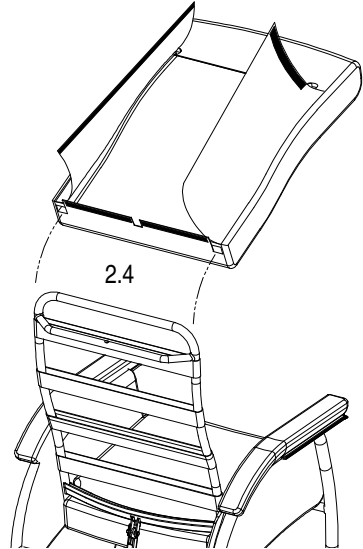
3. Seat Cushion Removal

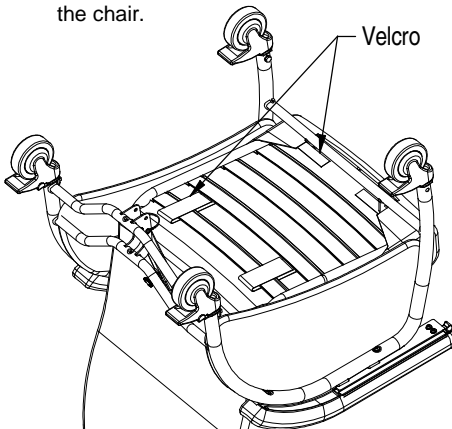- 3.1. Detach the Velcro strips located at the front and rear underneath the seat frame.
- 3.2. Pull on the seat cushion from the front of the chair.

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
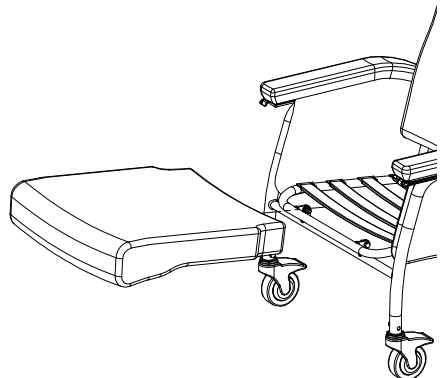
4. Seat Tilt Angle Adjustment (Z-25 model)

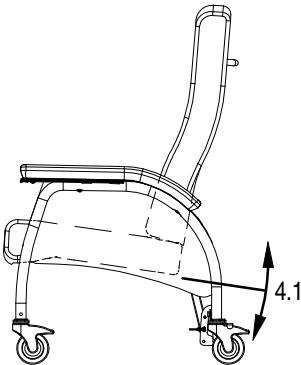- 4.1. Seat tilt angle adjustment (only available on Z-25 model) can be adjusted in 2 ways. **IMPORTANT:** the chair should be unoccupied.
- 4.2. Adjust the rear telescopic support. Unfasten the spring-loaded plunger by turning the handle counterclockwise until it stops (a) then pull on the plunger (b) while holding the seat with one hand. Adjust the support to the desired height and release the plunger making sure the plunger engages into a hole. **IMPORTANT:** fasten the plunger by tightly turning the handle clockwise.
- 4.3. For additional adjustment: rotate the wire retainer from the locking pin fastening the rear telescopic support to the chair base. While holding the telescopic support with one hand, pull the locking pin (c). Align the telescopic support with one of the three available holes on the chair base then insert the locking pin and secure it with the wire retainer.

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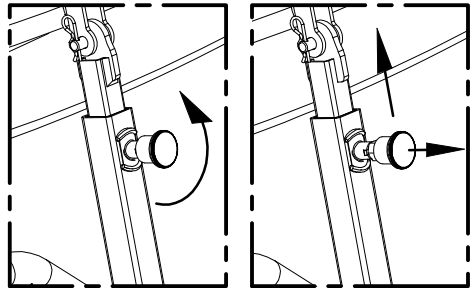
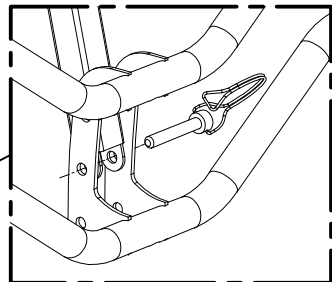
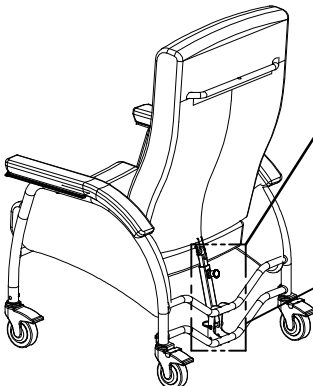
4.3c

5. Gas Spring Activated Seat Tilt Angle (Z-C2-A)

- 5.1. The Z-C2-A option provides tilt-in-space adjustment with a high lift angle (A) or low lift angle (B) to facilitate egress, or a backward tilt angle (C). The tilt angle can be adjusted by the chair occupant or by a caregiver.
- 5.2. Pull on the activator lever, adjust the tilt angle and release the lever.
- 5.3. For additional adjustment: the chair must be unoccupied while making this adjustment. While holding the gas spring with one hand, remove the locking pin fastening the gas spring to the base. Reposition the gas spring to the desired height and insert the locking pin into the base multi-hole fitting. Secure the locking pin with the wire retainer. Verify the tilt angle with the gas spring in extended and compressed position. Make sure the gas spring control cable is not pinched. Ask the patient to try the chair. Adjust again as needed.

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

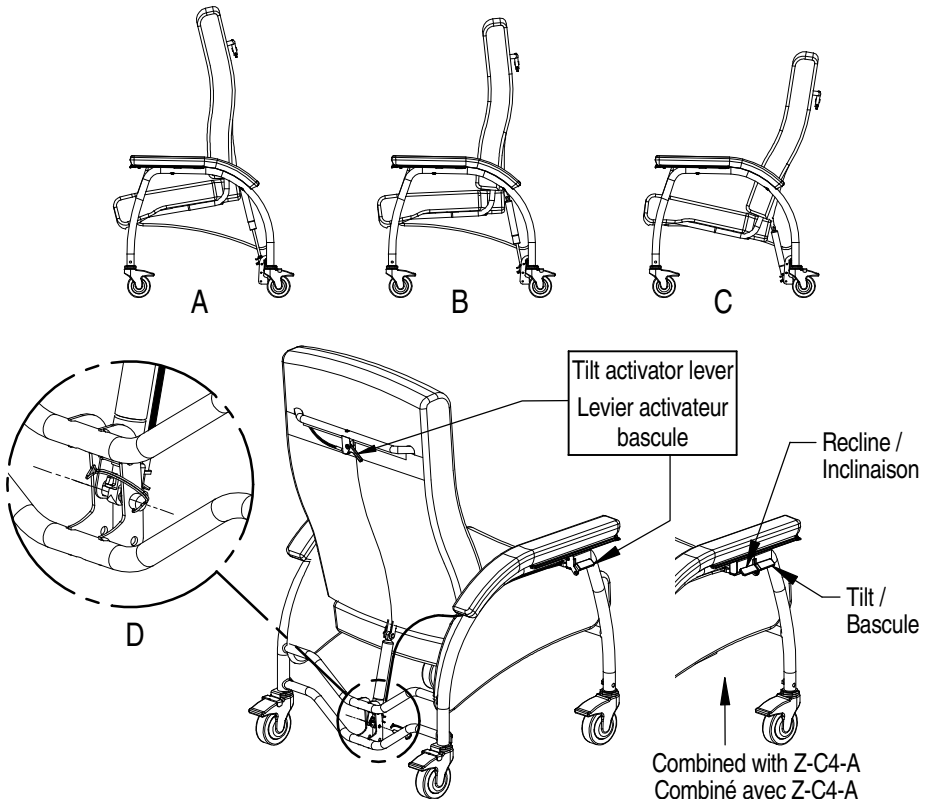
6. Seat Depth Adjustment (Z-C3-A and Z-C4-A)

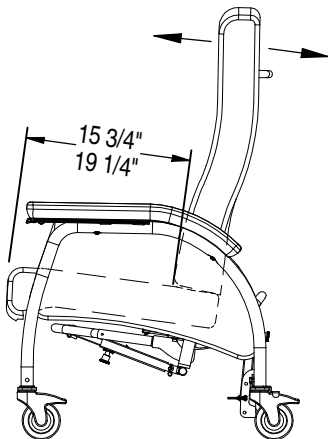- 6.1. Seat depth adjustment is accomplished by sliding the backrest forward or backward. The chair must be unoccupied while making adjustments. Behind the backrest and below the seat deck, locate the quick release levers (A).
- 6.2. Pull down the levers to loosen them (B). While holding the backrest sides with both hands, push or pull on the backrest to adjust seat depth. Make sure the backrest is straight.
- 6.3. Push the levers up firmly to fasten the backrest.

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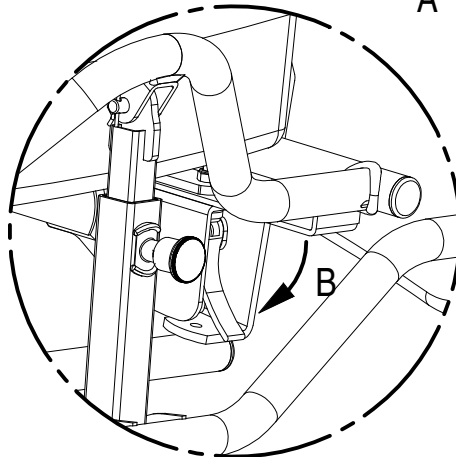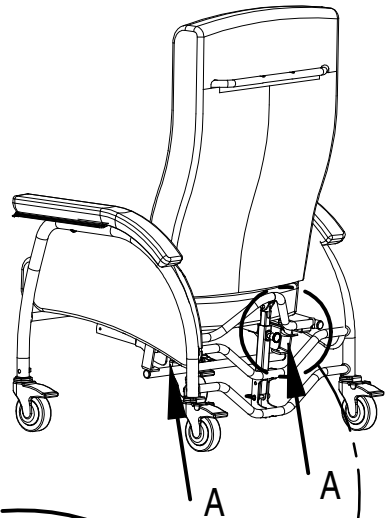
6.2


7. Backrest Recline Adjustment (Z-C3-A)

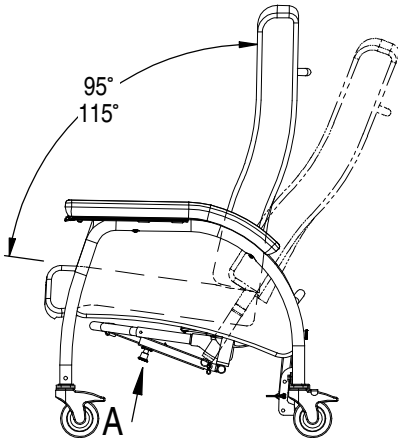- 7.1. Backrest recline adjustment is accomplished with a 4-position telescopic support placed underneath the seat deck (A). The chair must be unoccupied while making adjustments.
- 7.2. Unfasten the spring-loaded plunger by turning the handle counterclockwise until it stops (B) then pull on the plunger (C) while holding the backrest with one hand. Adjust backrest recline as required and release the plunger making sure the plunger engages into a hole.
- 7.3. **IMPORTANT:** fasten the plunger by tightly turning the handle clockwise.

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
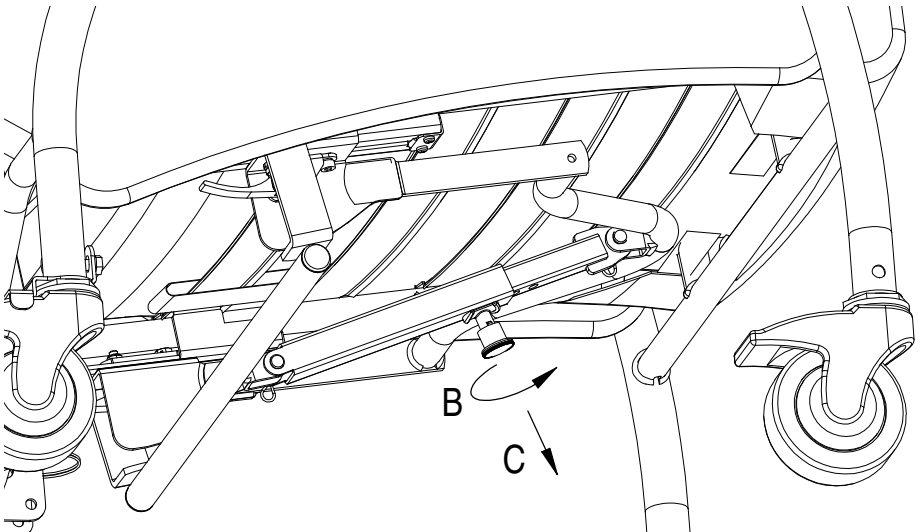
8. Gas Spring Activated Backrest Recline (Z-C4-A)

- 8.1. The Z-C4-A option provides backrest recline thanks to a gas spring located under the seat deck. The recline angle can be adjusted by the chair occupant or by a caregiver.
- 8.2. Pull on the activator lever, adjust the backrest recline and release the lever.

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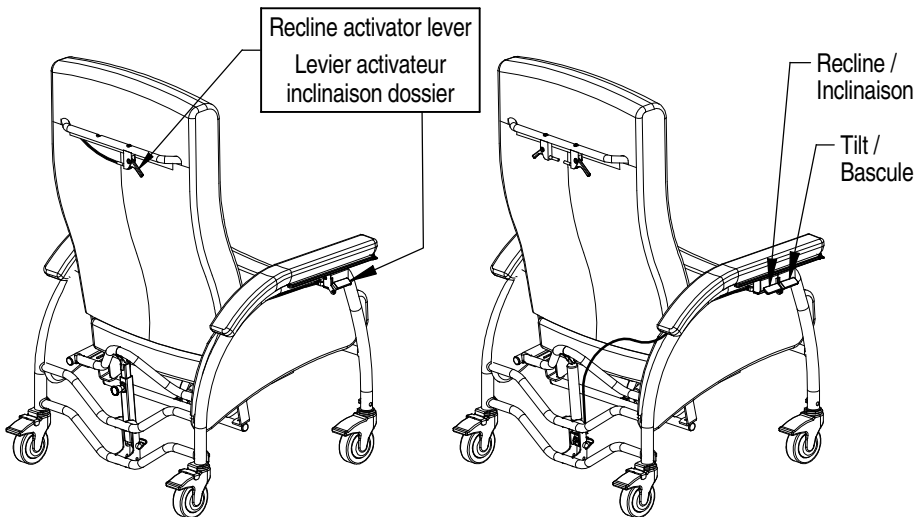
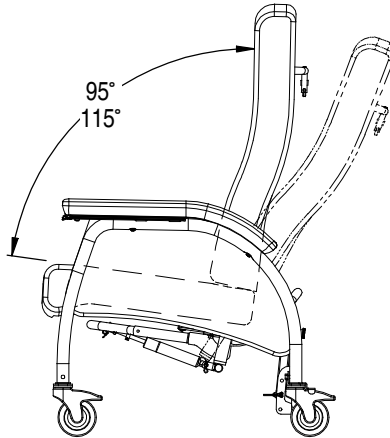
9. Adjustable Elevating Leg Rest (Z-C130L/R)

- 9.1. Remove the seat cushion as described in section n° 4.
- 9.2. Pull down the mounting bracket locking lever (a) then remove the mounting bracket from the elevating leg rest (b).
- 9.3. Install the mounting bracket underneath the seat deck, pressing it against the front tube. Insert two "U" bolts with two flat washers, two lock washers and two nuts; fasten the rear "U" bolt with two lock washers and two nuts. Do not tighten yet.
- 9.4. Insert the third "U" bolt on the front seat deck tube and fasten it with two lock washers, two nuts and two dome nuts. Firmly tighten all nuts.
- 9.5. Put back the seat cushion.
- 9.6. Refer to the elevating leg rest instructions for adjustment instructions.

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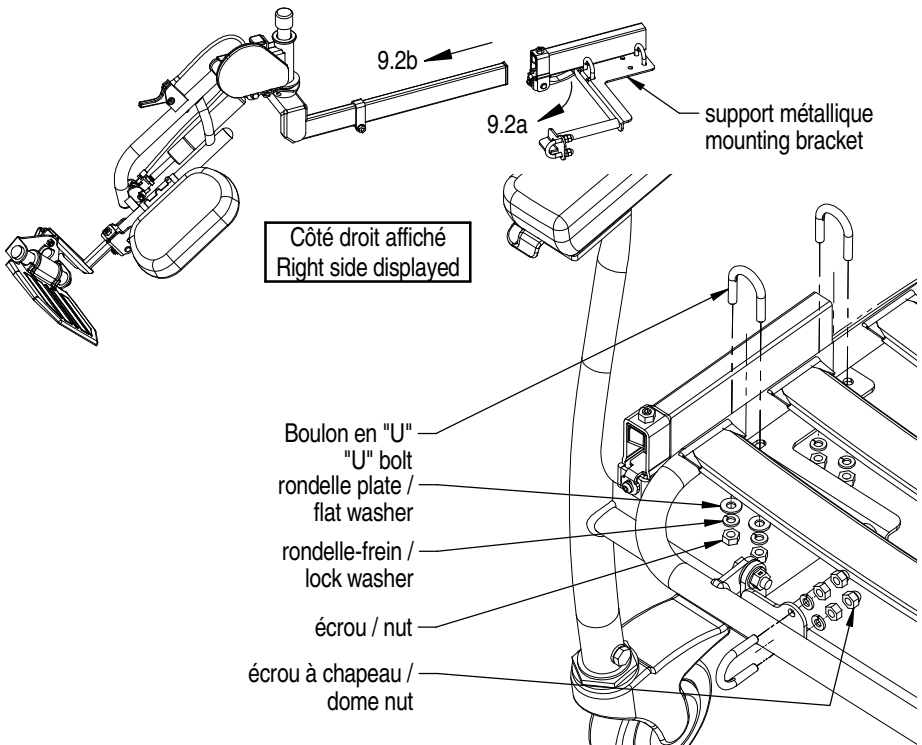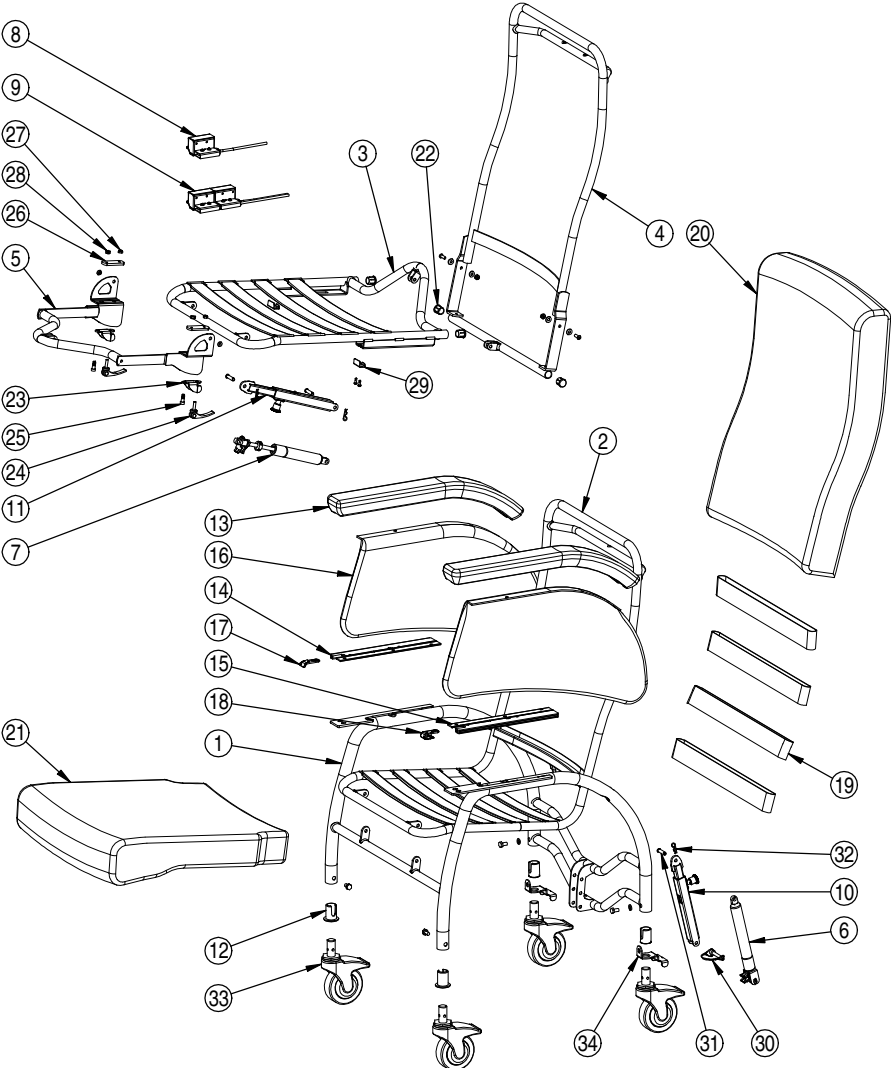* IMPORTANT: Always specify fabric color and pattern for all replacement part orders. The replacement parts may show some color difference with the original part. LPA Medical Inc. may not be held responsible for any such difference.

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TERMS & CONDITIONS

GENERAL:

- All items & prices listed are subject to change without notice. We reserve the right to change and/or improve any product as deemed advisable without notification. Shipments will be made at prevailing prices at time of shipment.

FREIGHT CLAIMS:

- LPA MEDICAL can not accept claims for merchandise that has been damaged in transit. Examine all merchandise upon delivery. If damage has occurred, it is the consignee's responsibility to file a claim with the carrier immediately if damage is apparent upon delivery or within 5 days if it is subsided damage.

RETURN GOODS

- Merchandise to be returned must have prior written authorisation from our Customer Service Department. All request most include the following information:
Reason for requested return

LPA MEDICAL invoice or acknowledgement number

LPA MEDICAL model or part number and number of cartons and items being returned.

Unauthorised returns will not be accepted. All authorised returns must be shipped pre-paid except where there has been an error in processing or a defect in materials or workmanship, in which case Customer Service will issue special instructions.

WARRANTY

- LPA MEDICAL warranty covers all defects in workmanship, except the exclusions listed in the not covered section below, for the following periods of time from date of purchase:

Lifetime for metal parts (frame): R-505, R-605, R-705, T-40, U-110, U-120, W-2, Z-15, Z-25 chairs

5 years for metal parts (frame): C-207, Optimum, O-07, R-505/605/705-SHD, S-150, S-250, Z-45 chairs

3 years only for auto lock glider mechanism

1 year for all other mechanism, gas spring cylinder or mechanical lock, control cable, wheels & casters, tray, upholstery.

If service is required under this warranty, contact our Customer Service Department in writing, giving the following information

- 1- Model or part number involved
- 2- Date of purchase, including LPA MEDICAL invoice or acknowledgement number
- 3- A description of the problem.

WHAT LPA MEDICAL WILL COVER

- LPA MEDICAL will at our option repair or replace at no charge to you, the original purchaser, the defective product or parts during the periods of given time

WHAT LPA MEDICAL WILL NOT COVER

- Normal wear and tears (frame, wheels, fabrics)
- Accidental damage including fire and water
- Fabric facing and metal oxidation (rust)
- Cost of transportation to or from LPA MEDICAL factory
- Unauthorised repairs made or attempted
- Damage resulting from abnormal or unreasonable use.

Your Rights Under Province or state law. This warranty gives you specific legal rights, you may also have other rights which vary from Province to Province or State to State.

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Adresse

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FOR YOUR RECORDS

**Purchaser's record for
LPA MEDICAL WARRANTY**

Date of purchase

Purchased from

Address

Model No, serial No. and quantity

Nomade II

Mobile Comfort Chair

Z-45 Model
with options

February 2015

Fauteuil de repos mobile

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Février 2015

LPA MÉDICAL INC.

2527 Avenue Dalton, Québec, Canada, G1P 3S6

Tel : (418) 681-1313, 800-663-4863 Fax : (418) 681-4488

lpa@lpamedical.com <http://www.lpamedical.com>