Thera-Glide®

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Modèles R-505, R-605, R-705

*Brevet U.S. # 6 406 095

**Brevet Can. #2 323 398

Therapeutic safety glider

Model R-505, R-605, R-705

*U.S. Patent # 6 406 095

**Can. Patent #2 323 398

Petit

Standard

Large

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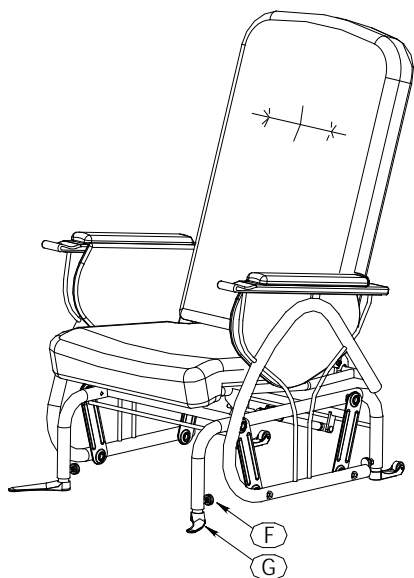
User's Guide

Your Thera-Glide® Safety Glider is built for years of trouble-free service. Thoroughly read and follow the instructions step by step to find out how to adjust, operate and care for your Thera-Glide Safety glider.

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
Using the Thera-Glide® Chair


For safest operation, please familiarize yourself with the following components:

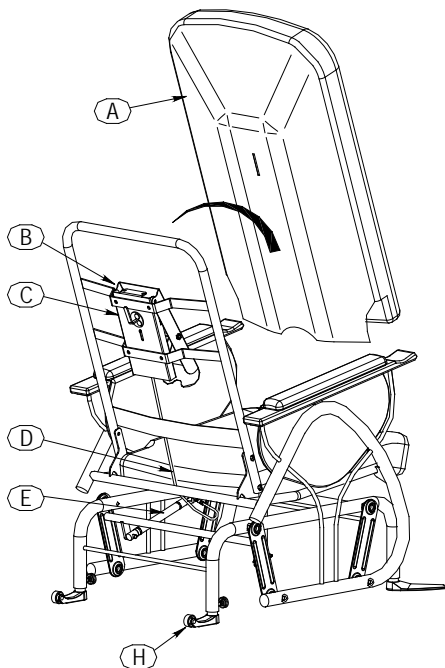- A**: Back cushion.
- B**: Backrest mechanism.
- C**: On/Off activator lever of locking system.
- D**: Locking system cable.
- E**: Locking cylinder.
- F**: Seat height adjustment.
- G**: Security anti-tips.
- H**: Rear wheels.

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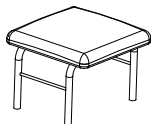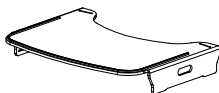
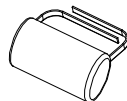
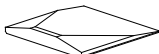
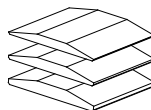
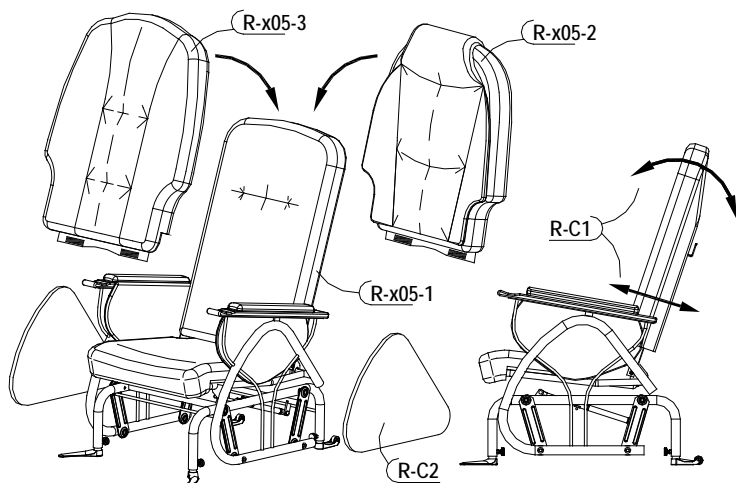
Models and Options

The Thera-Glide® chair is offered in different models with various options and accessories for a wide selection of comfort benefits.

R-x05-1: *Simplicity* cushion.

R-x05-2: *Classic* cushion.

R-x05-3: *Comfort +* cushion.

R-C1: Option for seat depth and back angle adjustment.

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WARNING

Be sure all adjustments are correct and bolts and nuts are tightened before sitting patient in the chair.

All adjustments must be supervised by a healthcare professional.

This product is intended for use in a secure environment and under constant supervision.

Thera-Glide®'s Maintenance

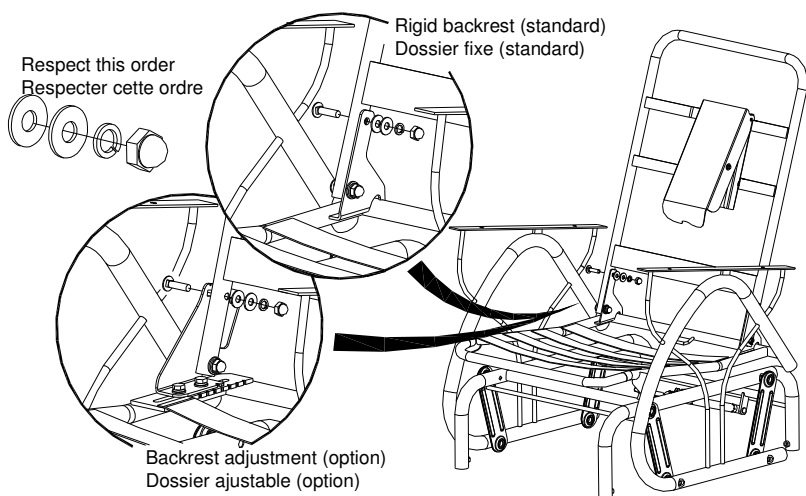
For best performance, it is necessary that the chair be kept clean and all parts adjusted properly.

When needed, the upholstered surfaces must be washed with a mild detergent diluted or dissolved in water. If necessary, a mild clear disinfectant can be used for hygienic purposes.

Metal parts of the chair should also be kept clean using the same mild detergent solution as above.

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
1. Fixed back metal structure assembly.

- 1.1. Place the backrest metal frame on the seat metal frame as illustrated.
- 1.2. Use the 5/16-18 x 1 1/4" carriage bolts, washer, lock-washer and cap nut to attach the backrest metal frame to the seat metal frame. Tighten bolts using wrench (not included).

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

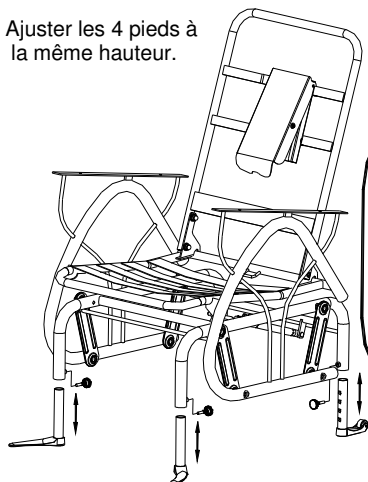
2. Adjustable Feet.

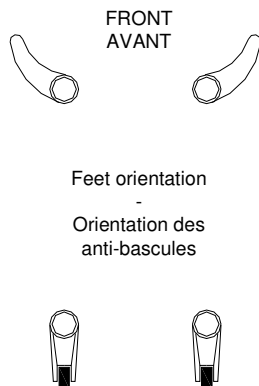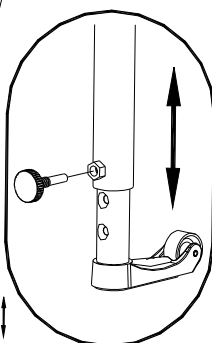- 2.1. Insert the adjustable feet as illustrated.
- 2.2. Slide the adjustable feet to the desired height. Align the holes, insert the tightening knob and tighten firmly.
- 2.3. Adjust the 4 feet to the same height.

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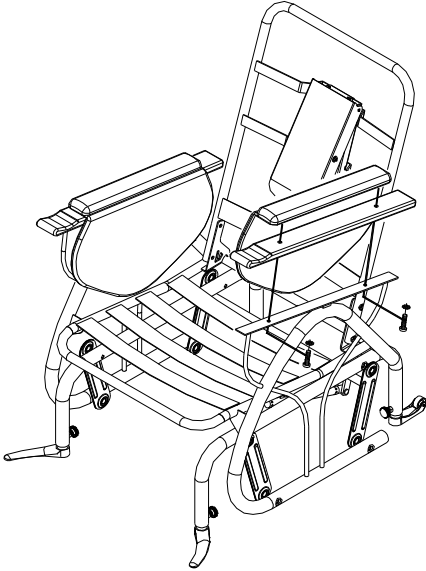
3. Armrest Assembly.

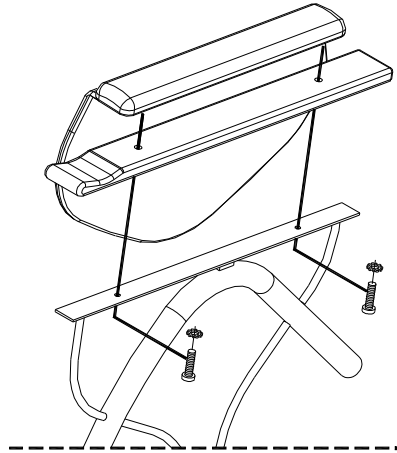- 3.1. Place the wooden armrest and the armrest upholstery on the metal frame as illustrated.
- 3.2. Fix the machine screw $\frac{1}{4}$ -20 x $1\frac{1}{2}$ " (38mm), with tooth lock washer, through the metal frame hole and wooden armrest to secure the padded armrest. Tighten firmly.

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

4. Seat Assembly.

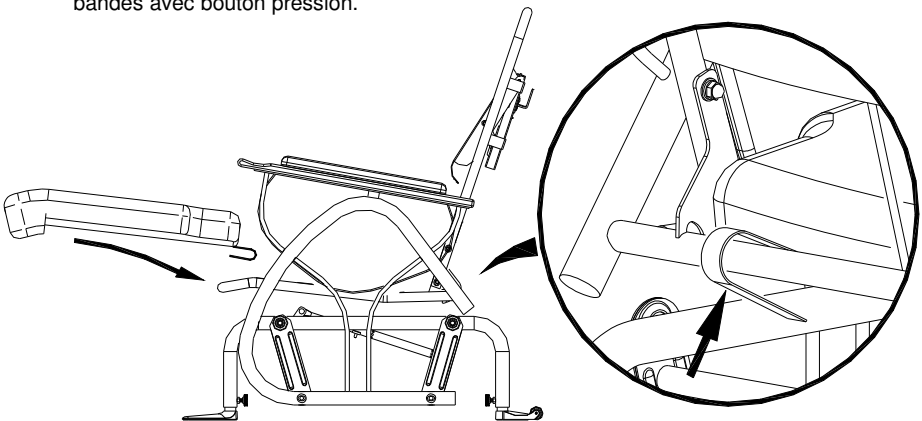- 4.1. Slide the seat apron's pocket to the front tube and slip the cushion to the back. Secure the cushion with velcro around the tube. Seal the front velcro.

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
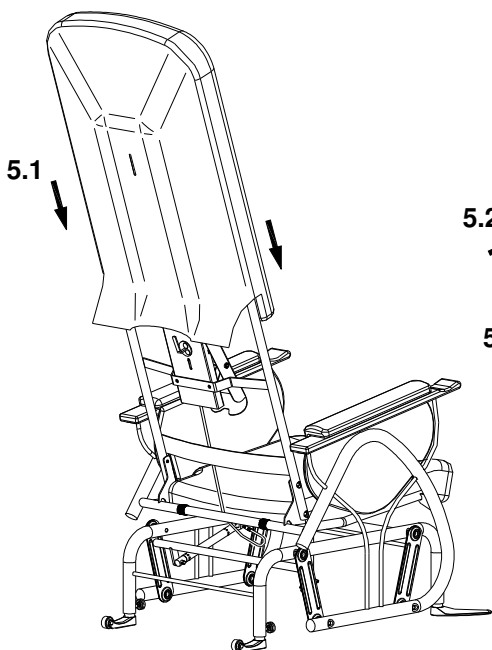
5. Back Cushion Assembly.

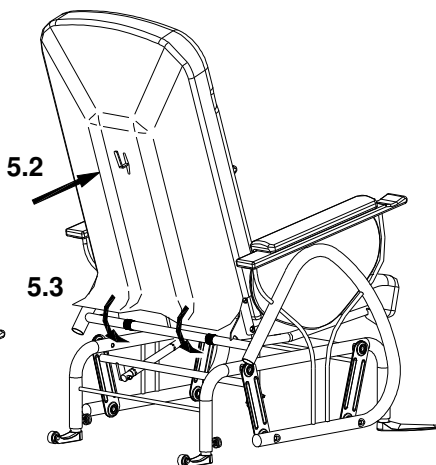- 5.1. Slip the backrest cushion over the backrest.
- 5.2. Allow the glider manual locking lever to pass through in the back cushion hole.
- 5.3. Secure lower upholstery of backrest cushion to chair with velcro.

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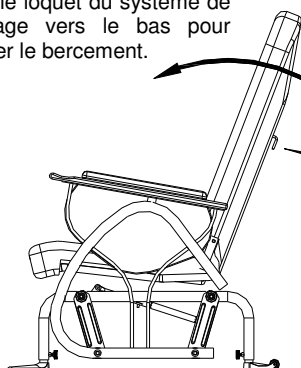
6. Lock System.

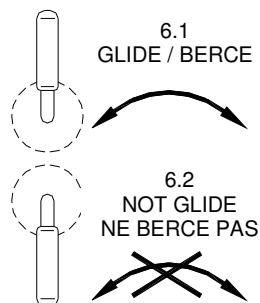- 6.1. To unlock the glide function, rotate the locking lever up.
- 6.2. To lock the glide function, rotate the locking lever down.

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
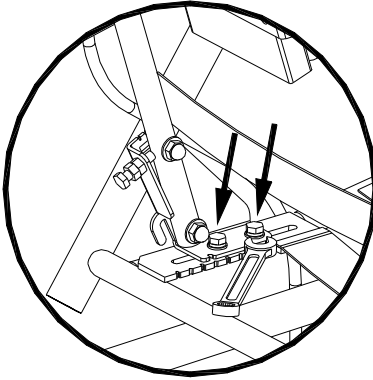
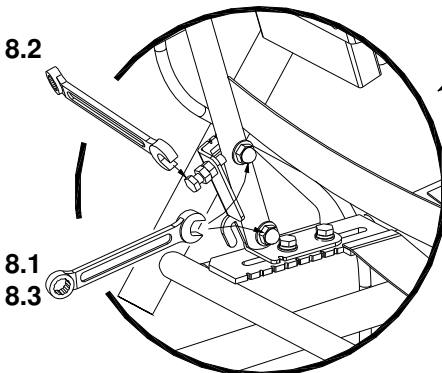
Options

Seat Depth and Backrest Angle Adjustment.


7. Seat Depth Adjustment.

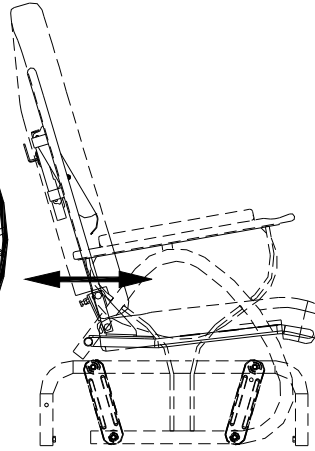
- 7.1. On each rear side of the seat loosen (DO NOT REMOVE) the 2 sliding bolts.
- 7.2. Adjust the seat depth correctly and tighten the bolts. Adjust each side to the same depth.

8. Backrest Angle Adjustment.

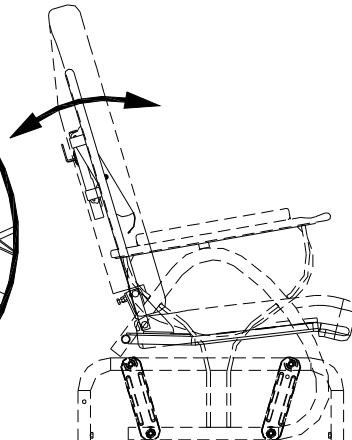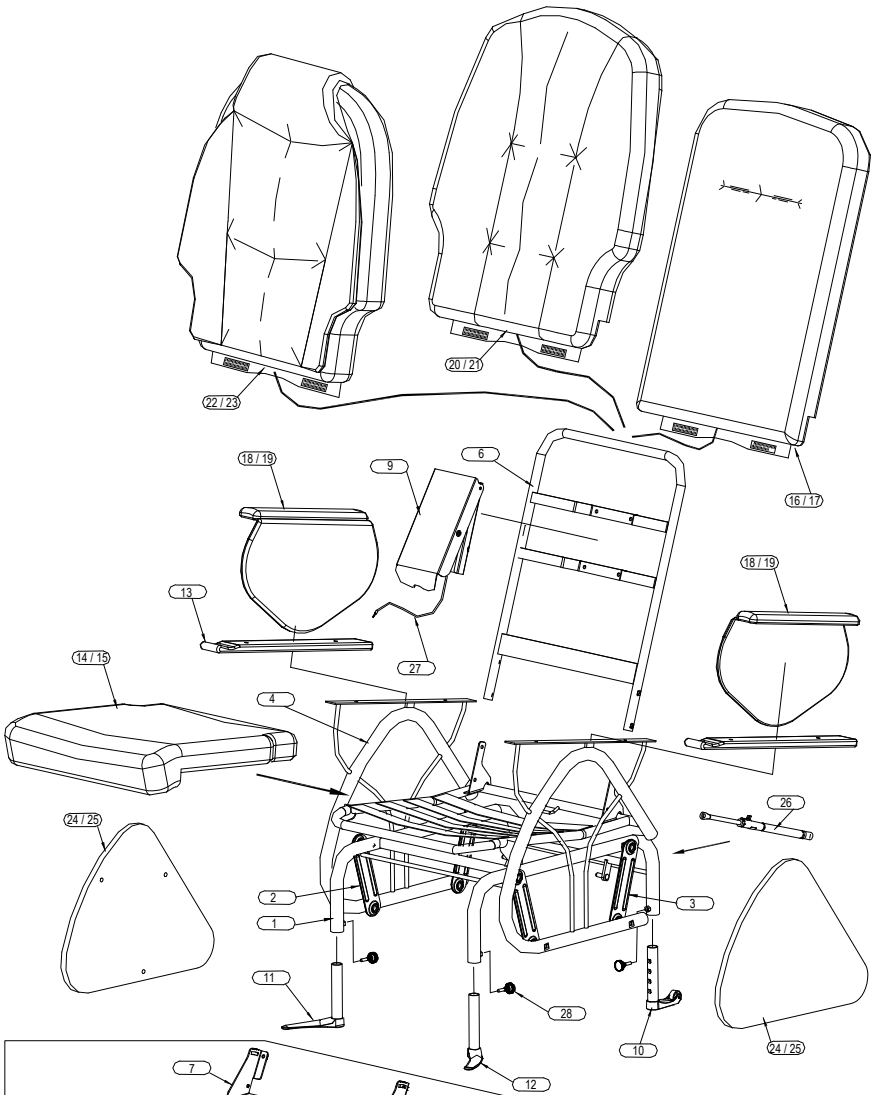
- 8.1. On each rear side of the seat loosen (DO NOT REMOVE) the 2 bolts.
- 8.2. Tighten the rear bracket bolts.
- 8.3. Re-tighten the bolts (loosened in 8.1.)

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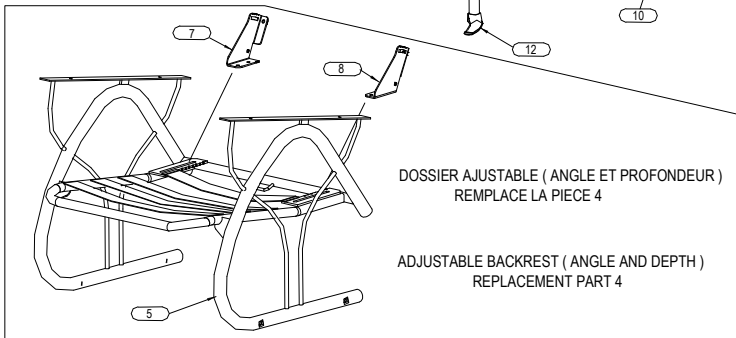ADJUSTABLE BACKREST (ANGLE AND DEPTH)
REPLACEMENT PART 4

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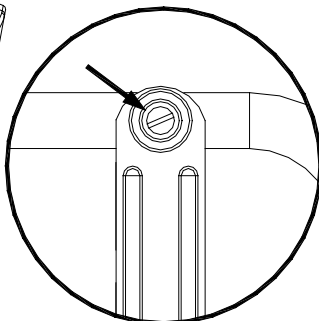
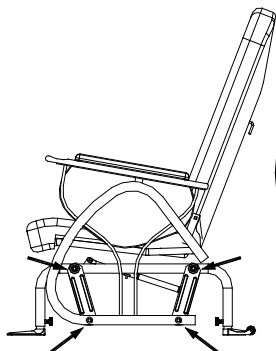
Available with the backrest adjustment option

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Always specify fabric color and pattern, wood stain, and the Thera-Glide®'s serial number with all replacement part orders. Replacement parts may exhibit color differences from the original part. LPA Medical will not be responsible of any possible difference.

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Noisy Mechanism

There may be dried paint between the screw nut and the glide bearing. Remove any excess paint using the tip of a needle and add lubricant/oil sparingly.

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TERMS & CONDITIONS

GENERAL:

• All items & prices listed are subject to change without notice. We reserve the right to change and/or improve any product as deemed advisable without notification. Shipments will be made at prevailing prices at time of shipment.

FREIGHT CLAIMS:

• LPA MEDICAL cannot accept claims for merchandise that has been damaged in transit. Examine all merchandise upon delivery. If damage has occurred, it is the consignee's responsibility to file a claim with the carrier immediately if damage is apparent upon delivery or within 5 days if it is subsidized damage.

RETURN GOODS

• Merchandise to be returned must have prior written authorisation from our Customer Service Department. All request most include the following information:

Reason for requested return

LPA MEDICAL invoice or acknowledgement number

LPA MEDICAL model or part number and number of cartons and items being returned.

Unauthorised returns will not be accepted. All authorised returns must be shipped pre-paid except where there has been an error in processing or a defect in materials or workmanship, in which case Customer Service will issue special instructions.

WARRANTY

• LPA MEDICAL warranty covers all defects in workmanship, except the exclusions listed in the not covered section below, for the following periods of time from date of purchase:

Lifetime for metal parts (frame): R-505, R-605, R-705, T-40, U-110, U-120, W-2, Z-15, Z-25 chairs

5 years for metal parts (frame): C-207, Optimum, O-07, R-505/605/705-SHD, S-150, S-250, Z-45 chairs

3 years only for auto lock glider mechanism

1 year for all other mechanism, gas spring cylinder or mechanical lock, control cable, wheels & casters, tray, upholstery.

If service is required under this warranty, contact our Customer Service Department in writing, giving the following information

- 1- Model or part number involved
- 2- Date of purchase, including LPA MEDICAL invoice or acknowledgement number
- 3- A description of the problem.

WHAT LPA MEDICAL WILL COVER

• LPA MEDICAL will at our option repair or replace at no charge to you, the original purchaser, the defective product or parts during the periods of given time

WHAT LPA MEDICAL WILL NOT COVER

- Normal wear and tears (frame, wheels, fabrics)
- Accidental damage including fire and water
- Fabric fading and metal oxidation (rust)
- Cost of transportation to or from LPA MEDICAL factory
- Unauthorised repairs made or attempted
- Damage resulting from abnormal or unreasonable use.

Your Rights Under Province or state law. This warranty gives you specific legal rights, you may also have other rights which vary from Province to Province or State to State.

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**Purchaser's record for
LPA MEDICAL WARRANTY**

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Model No, serial No., description and quantity

Thera-Glide®

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Modèles R-505, R-605, R-705

*Brevet U.S. # 6 406 095

**Brevet Can. #2 323 398

Therapeutic safety glider

Model R-505, R-605, R-705

*U.S. Patent # 6 406 095

**Can. Patent #2 323 398

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Update: April 16th 08

LPA MÉDICAL INC.

2527 Avenue Dalton, Québec (Qc), Canada, G1P 3S6

Tel: (418) 681-1313, (800) 663-4863

Fax : (418) 681-4488

lpamedical.com

<http://www.lpamedical.com>

OPTIMA PRODUCTS INC.

2230 Edgewood Avenue – STE 3A,

St. Louis Park, MN 55426 USA

Tel: (952) 745-0027

Toll free: 1-888-990-9394

Fax: (952) 745-0197

optima@optimaol.com

<http://www.optimaol.com>