Thera-Glide™

Safety Glider

T-40XX-22 Transitional Model

*U.S. Patent # 6 406 095

**Can. Patent # 2 323 398

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*Brevet U.S. # 6 406 095

**Brevet Can. # 2 323 398

User's Guide

Your Thera-Glide™ Safety Glider is built for years of comfort and trouble free service. Thoroughly read and follow the instructions step by step to find out how to adjust, operate and care for your Thera-Glide™ Safety Glider.

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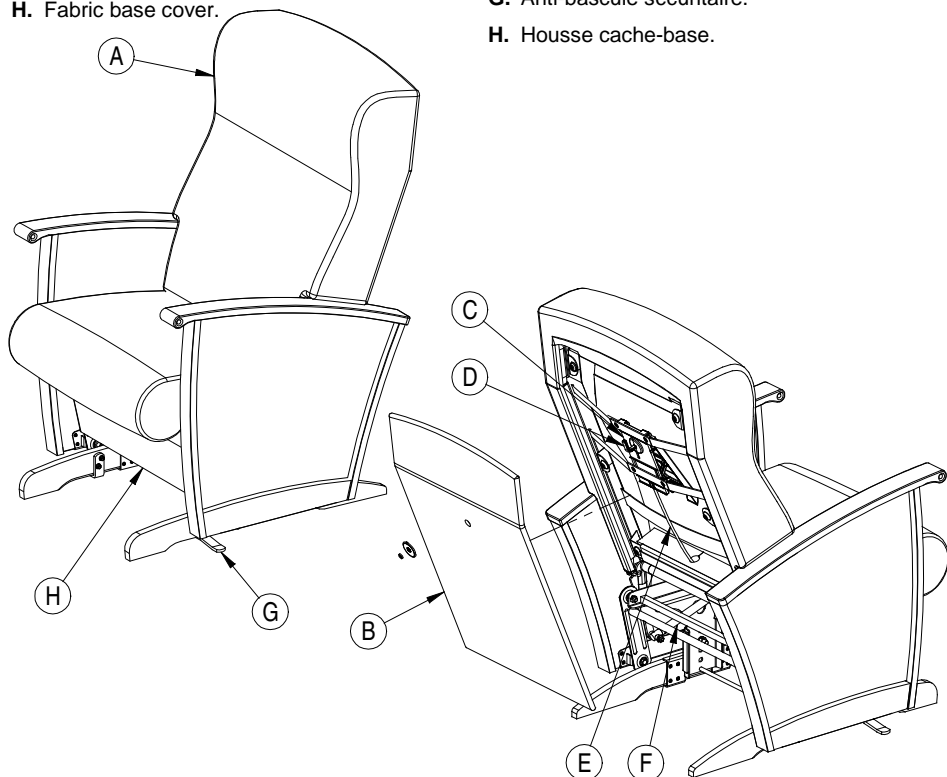
Using the Thera-Glide Chair™

For safest operation, please familiarize yourself with the following components:

- A. Back cushion.
- B. Removable backrest panel.
- C. Backrest mechanism.
- D. Locking system On/Off activator.
- E. Locking system cable.
- F. Locking cylinder.
- G. Anti-tips for your security.
- H. Fabric base cover.

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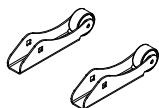
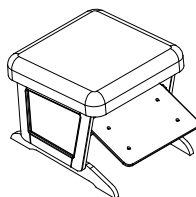
WOF-22 Fixed wood ottoman /
Tabouret en bois fixe

WARNING

Be sure all adjustments are correct and bolts and nuts are tightened before sitting patient in the chair.

All adjustments must be supervised by a healthcare professional.

This product is intended for use in a secure environment under supervision.

Wood Thera-Glide™'s maintenance

For best performance, it is necessary that the chair is kept clean and all parts adjusted properly.

When needed, the upholstered surfaces must be washed with a mild detergent diluted or dissolved in water. If necessary, a mild clear non-chlorinated disinfectant can be used for hygienic purposes.

Metal parts of the chair should also be kept clean using the same mild detergent solution as above.

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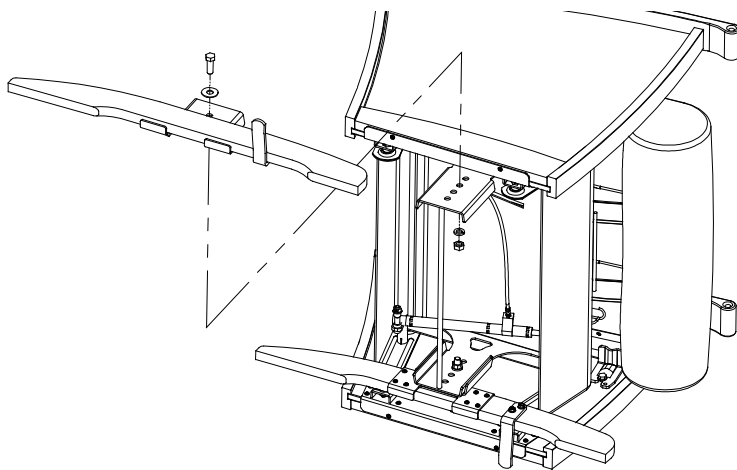
1. Height Adjustment.

- 1.1. Tip the chair on its side. Using one 5/8" wrench and one 11/16" wrench, unfasten the hexagonal nut on the leg stand inner side, and remove the washers and the bolt. If the leg stand seems stuck, carefully twist it to set it free. Move it to the desired height, then replace the bolt, washers and hex nut and tighten firmly.
- 1.2. Repeat on the chair's other side.

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
2. Locking System


- 2.1. To unlock the chair's glide motion, rotate the locking lever located on the back of the chair upward and clockwise.
- 2.2. To lock the chair's glide motion, rotate the locking lever downward and counter-clockwise.

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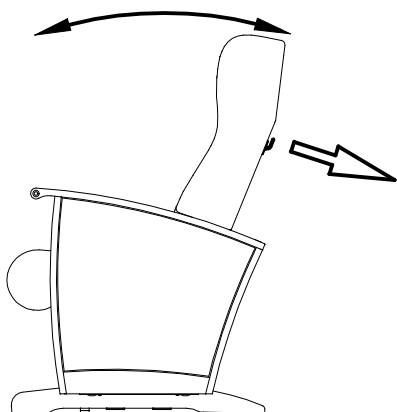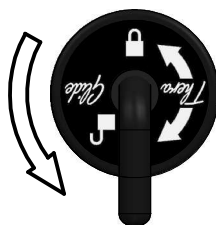
2.2
Does not glide /
Ne berce pas


3. Upholstered Back Panel Removal

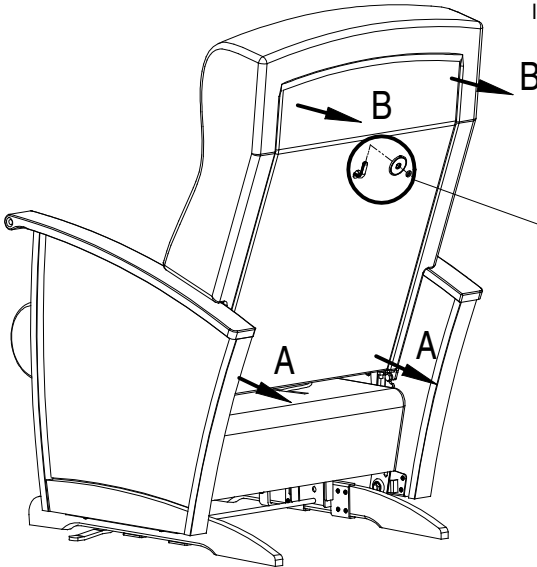- 3.1 First remove the O-ring and the washer from the locking lever.
- 3.2 Remove the rear back panel by pulling first on the two bottom corners (A), then on the two top corners (B). Take care not to damage the hole in the back panel when going through the locking lever.

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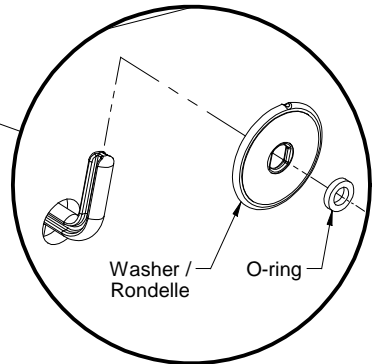
3.1

4. Seat Cushion Removal

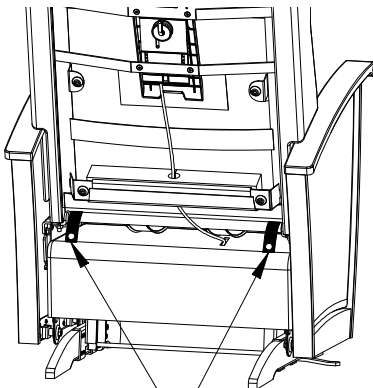
- 4.1 Detach the rear straps.
- 4.2 Pull on the front to remove the seat cushion.

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
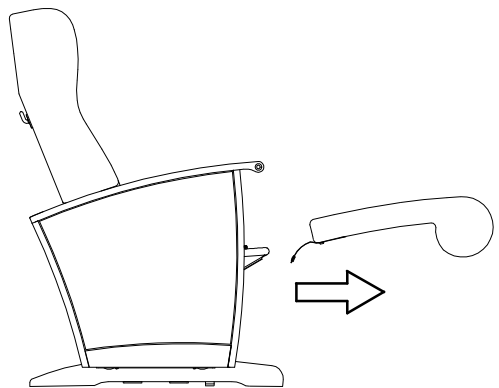
4.2

5. Backrest Frame Removal

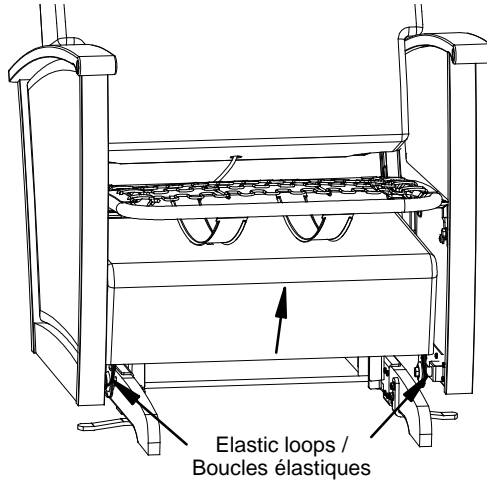- 5.1 To get access to the locking cylinder, partly lift up the fabric base cover by untying the elastic loops at the bottom front.

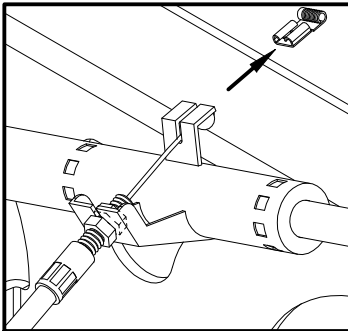- 5.2 Remove the metal cable closure from the locking lever.
- 5.3 Using either two 10mm wrenches or one 10mm wrench and a flat screwdriver, unscrew the hexagonal nuts holding the conduit cable threaded end to the locking cylinder holder.
- 5.4 Remove the cable from the locking cylinder and thread it out of the slot in the fabric base cover.

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
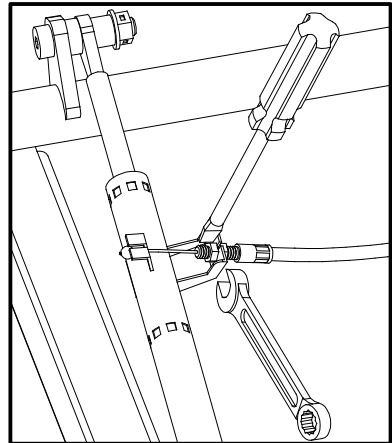
5.3


5.5 Unfasten the 4 hexagonal nuts with a 7/16" wrench to remove the backrest frame. Take care not to damage the chair's sides when removing the backrest frame.

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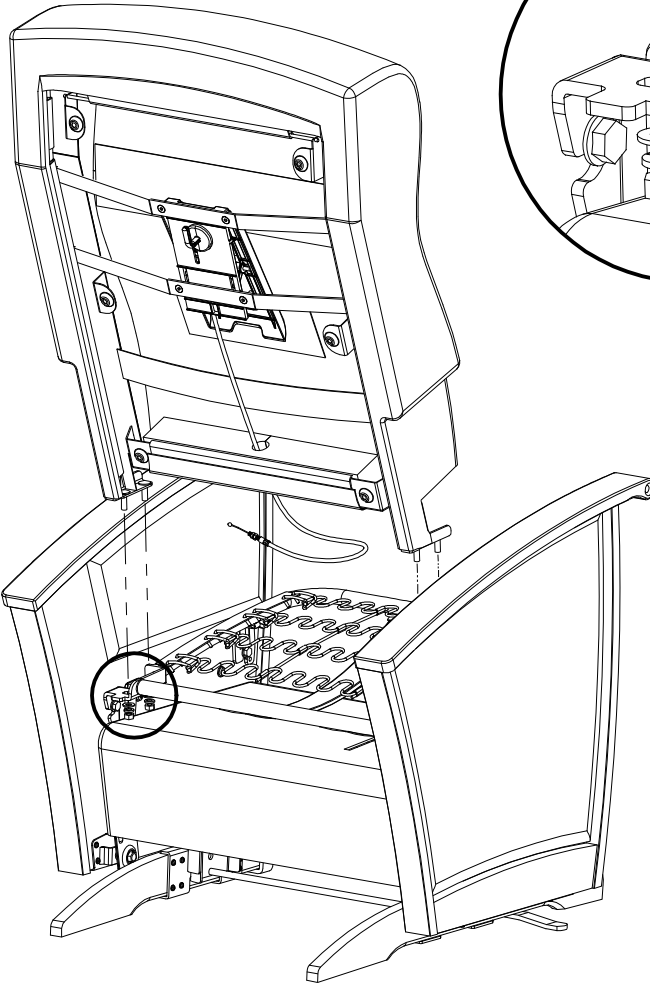6. Backrest Cover Removal

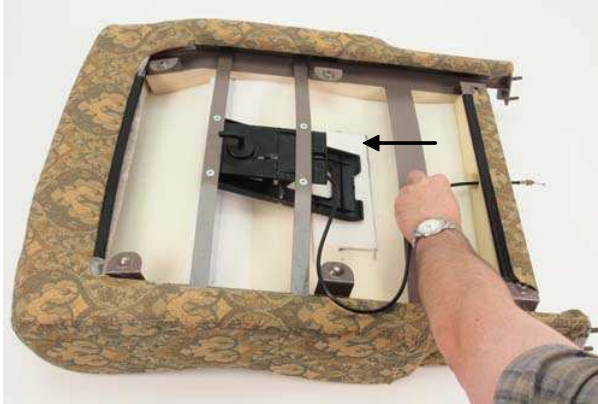- 6.1 Place the backrest frame on a table, with the rear up. Remove the cable from the hole in the bottom foam block and in the backrest cover (6.1A) then remove the bottom foam block (6.1B).

6.1A

6.1B

- 6.2 Pull on the black J-welt to remove it from the bottom metal cross-section (fig. 6.2). Compressing the foam helps by loosening the backrest cover.

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- 6.3 Pull on the black J-welt at the top to remove it from the steel rod.
- 6.4 Remove the black J-welt from the metal side frame on each side of the backrest metal frame.

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6.3

6.4

- 6.5 Rotate and insert the plastic pull plate into the slot in the foam (fig. 6.5A) then pull it completely through from the front of the backrest frame (fig. 6.5B). The backrest cover can now be completely removed.

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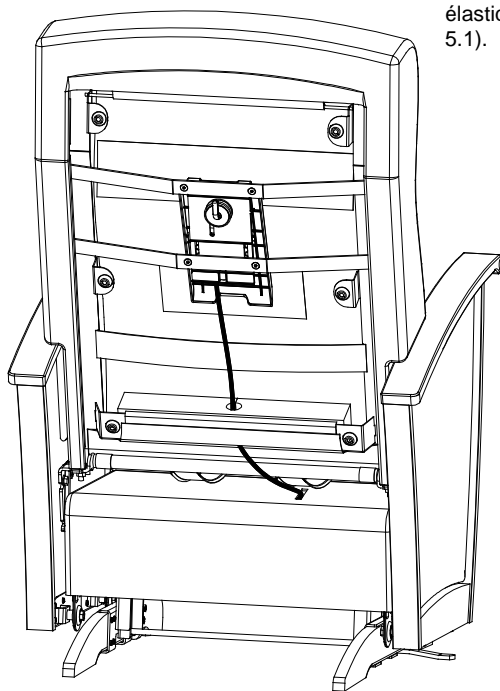
6.5B

7. Cable installation / adjustment

Note: for Thera-Glide™ chairs delivered fully assembled, the cable tension is factory adjusted with the correct amount of slack.

- 7.1. Attach the activator control cable to the cylinder. But first make sure the cable passes through the hole in the backrest bottom foam block and the fabric cover then behind the seat frame rear cross tube, and through the fabric base cover as shown in fig 7.1. To make the next steps easier, partly remove the fabric base cover by untying the elastic loops at the front of the chair (refer to 5.1).

- 7.2. Place the threaded end of the conduit cable in its holder on the locking cylinder and fasten the loose hexagonal cylinder and fasten the loose hexagonal nut to secure it. Initially set the hexagonal nut on the conduit side so 2 to 3 threads are showing. Adjust later as needed. Slide the steel wire with the ball end through the slot of the locking lever. While pulling on the steel wire, adjust the hexagonal nuts so that the ball end sticks out by half from the tip of the locking lever slot. **Note:** make sure the backrest mechanism plate is not depressed.

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
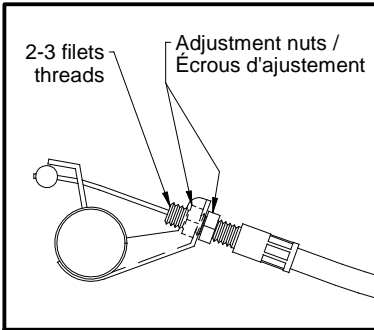
- 7.3 Insert the metal cable closure on the locking lever (A-7). When the metal cable closure is inserted, push it in so that it completely covers the lever's slot.
- 7.4 Manually operate the mechanism to ensure that all is working correctly. If needed, adjust the adjustment nuts. When it's correctly adjusted, use a flat screwdriver to secure the set nut on the end side of the cable conduit, and use a 10mm wrench to tighten the adjustment nut on the opposite side. **Be sure that the nuts are tightened enough.**

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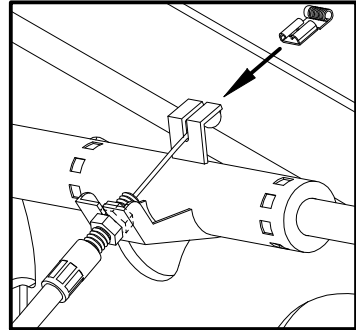
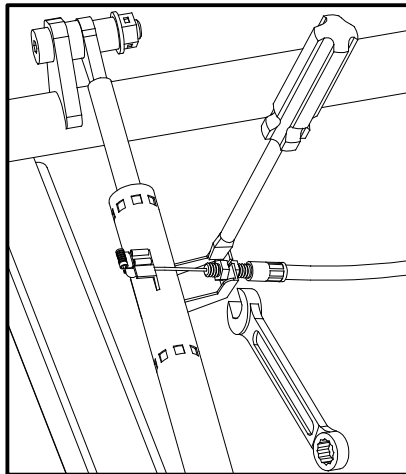
7.3

7.4

- 7.5 Put the backrest cushion back on the chair and manually operate the mechanism again to ensure it is working correctly. Readjust if needed.
- 7.6 To finish, apply a very small dab of general usage grease on the steel wire in the slot of the locking lever.

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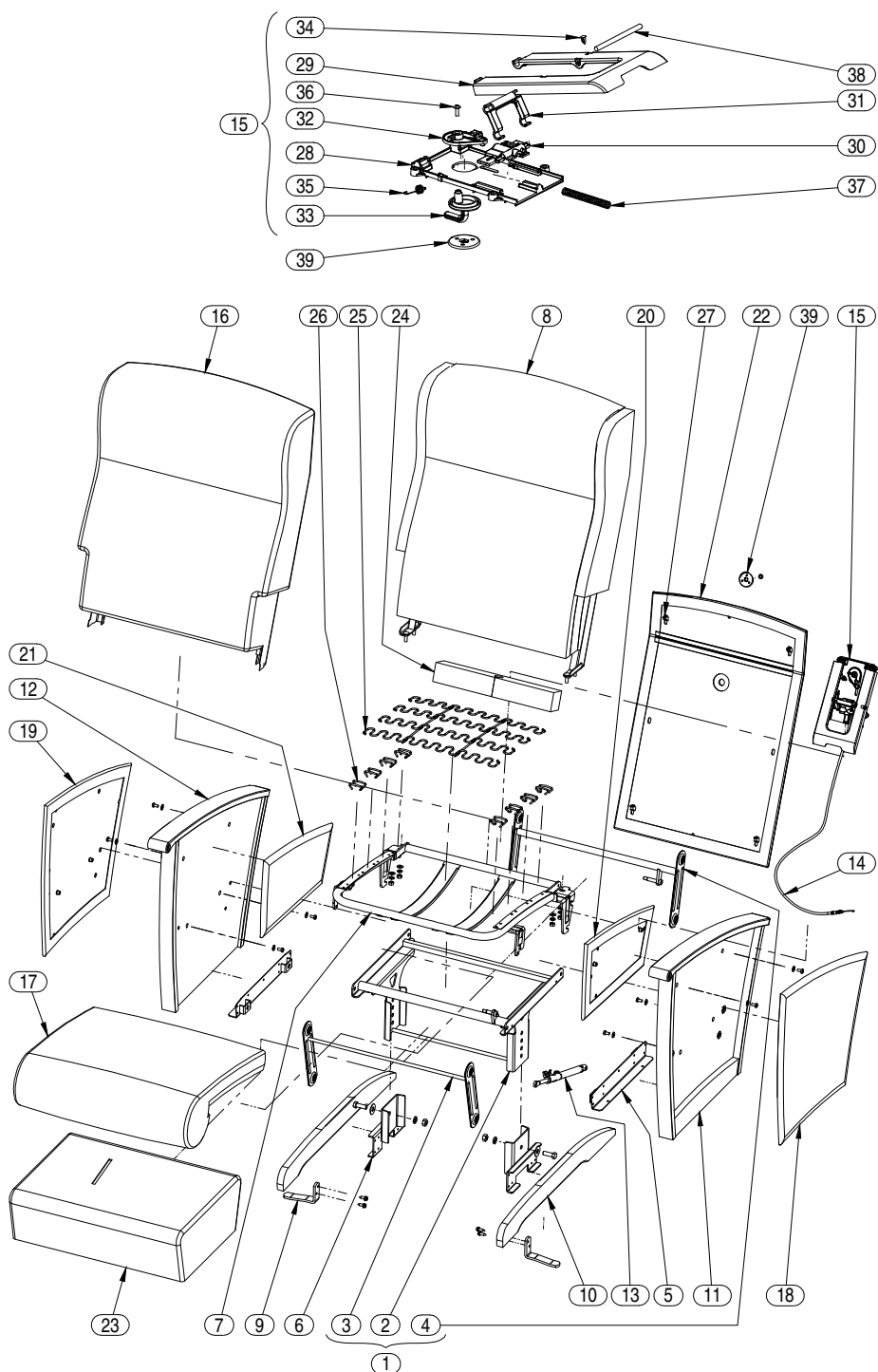Important: Note "C"

Always specify fabric color and pattern, wood stain and the Thera-Glide™'s serial number for all replacement parts order. Replacement parts could exhibit minor color/stain variances from the original part. LPA Medical will not be responsible of any possible variances.

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TERMS & CONDITIONS

GENERAL:

- All items & prices listed are subject to change without notice. We reserve the right to change and/or improve any product as deemed advisable without notification. Shipments will be made at prevailing prices at time of shipment.

FREIGHT CLAIMS:

- LPA MEDICAL can not accept claims for merchandise that has been damaged in transit. Examine all merchandise upon delivery. If damage has occurred, it is the consignee's responsibility to file a claim with the carrier immediately if damage is apparent upon delivery or within 5 days if it is subsided damage.

RETURN GOODS

- Merchandise to be returned must have prior written authorisation from our Customer Service Department. All request most include the following information: Reason for requested return

LPA MEDICAL invoice or acknowledgement number

LPA MEDICAL model or part number and number of cartons and items being returned.

Unauthorised returns will not be accepted. All authorised returns must be shipped pre-paid except where there has been an error in processing or a defect in materials or workmanship, in which case Customer Service will issue special instructions.

WARRANTY

- LPA MEDICAL warranty covers all defects in workmanship, except the exclusions listed in the not covered section below, for the following periods of time from date of purchase:

Lifetime for metal parts (frame): R-505, R-605, R-705, T-40, U-110, U-120, W-2, Z-15, Z-25 chairs

5 years for metal parts (frame): C-207, Optimum, O-07, R-505/605/705-SHD, S-150, S-250, Z-45 chairs

3 years only for auto lock glider mechanism

1 year for all other mechanism, gas spring cylinder or mechanical lock, control cable, wheels & casters, tray, upholstery.

If service is required under this warranty, contact our Customer Service Department in writing, giving the following information

- 1- Model or part number involved
- 2- Date of purchase, including LPA MEDICAL invoice or acknowledgement number
- 3- A description of the problem.

WHAT LPA MEDICAL WILL COVER

- LPA MEDICAL will at our option repair or replace at no charge to you, the original purchaser, the defective product or parts during the periods of given time

WHAT LPA MEDICAL WILL NOT COVER

- Normal wear and tears (frame, wheels, fabrics)
- Accidental damage including fire and water
- Fabric facing and metal oxidation (rust)
- Cost of transportation to or from LPA MEDICAL factory
- Unauthorised repairs made or attempted
- Damage resulting from abnormal or unreasonable use.

Your Rights Under Province or state law. This warranty gives you specific legal rights, you may also have other rights which vary from Province to Province or State to State.

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Adresse

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FOR YOUR RECORDS

**Purchaser's record for
LPA MEDICAL WARRANTY**

Date of purchase

Purchased from

Address

Model No, serial No. and quantity

Thera-Glide™

Safety Glider

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T-40XX-22 Transitional Model

*U.S. Patent # 6 406 095

**Can. Patent # 2 323 398

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*Brevet U.S. # 6 406 095

**Brevet Can. # 2 323 398

LPA MÉDICAL INC.

2527 Avenue Dalton, Québec, Canada, G1P 3S6

Tel : (418) 681-1313, 800-663-4863 Fax : (418) 681-4488

lpa@lpamedical.com <http://www.lpamedical.com>

OPTIMA PRODUCTS INC.

2230 Edgewood Avenue - STE 3A,

St. Louis Park, MN 55426 USA

Tel: (888) 990-9394 Fax: (952) 745-0197

<http://optimaol.com>